



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59281 (06/2018)

| | |
|---|--|
| Serial Number 80-006678 | Instrument Location Grand Forks PD |
| Reason for Install/Repair | |
| <input type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input checked="" type="checkbox"/> Other (Specify) Maintenance conducted at UND PD | |

Check When Done:

- 1. Surge Protector Installed/Property Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. Radio Frequency Interference (RFI) Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if any): **Maintenance remotely performed by Narehood.**
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN504096, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

| | |
|--|---------------------------|
| Field Inspector Signature [Signature] 653 | Date 11/12/2024 |
|--|---------------------------|

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

| | |
|--|--------------------------------------|
| Reviewed/Certified By [Signature] Anna Narehood | Certified Date 18 Nov 2024 |
|--|--------------------------------------|

**Updated Grand Forks PD operator file remotely by UND connection.
 -AEN 11/18/24**

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006678
Location = GFPD 8164.14.00 09/16
11/12/2024 14:22

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-|=|
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-|=|
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$\$%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 01923080A3
Standard Cyl #? 57
Standard Expiration? 02/05/2025
Oper No? 131747

Flow Cal. Date: 08/22/2016
Slope 686
Intercept -611596

IR Calibration Date: 08/22/2016

| | 3um | 9um |
|--------------------|--------|--------|
| 0th Coef(*100): | -15445 | -21650 |
| 1st Coef(*100): | 251854 | 131766 |
| 2nd Coef(*100): | 3094 | 1378 |
| H2O adj(mg/l*10k): | 549 | 513 |

***** Printer Test End *****

 653

Operator Signature
ADAM SOLAR

Remarks:

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006678
Location = GFPD 8164.14.00 09/16
11/12/2024 14:23

DRY CAL CHECK

| Test | AC | Time |
|-------------|-------|-------|
| 01 Room Air | 0.000 | 14:23 |
| 02 Std. Gas | 0.082 | 14:24 |
| 03 Room Air | 0.000 | 14:24 |
| 04 Std. Gas | 0.082 | 14:25 |
| 05 Room Air | 0.000 | 14:25 |
| 06 Std. Gas | 0.082 | 14:26 |
| 07 Room Air | 0.000 | 14:26 |

Lot No = 01923080A3
Cyl No = 57
Exp Date = 02/05/2025
County = 18 Oper No. = 131747



Operator Signature
ADAM SOLAR

Remarks:

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

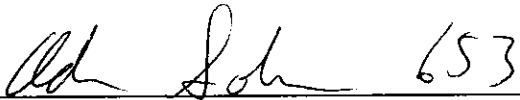
CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006678
Location = GFPD 8164.14.00 09/16
11/12/2024 14:27

DRY CAL CHECK

| Test | AC | Time |
|-------------|-------|-------|
| 01 Room Air | 0.000 | 14:28 |
| 02 Std. Gas | RFI* | 14:28 |
| 03 Room Air | 0.000 | 14:29 |

*Invalid Test
Inhibited - RFI

Lot No = 01923080A3
Cyl No = 57
Exp Date = 02/05/2025
County = 18 Oper No. = 131747

 653

Operator Signature
ADAM SOLAR

Remarks:

Form 106-I8000