



ETHANOL GAS STANDARD CYLINDER REPORT

NORTH DAKOTA OFFICE OF ATTORNEY GENERAL

CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM

SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <i>Thompson, Chad</i>		
Location <i>Cass Jail - Fargo</i>	Intoxilyzer® Serial Number <i>80-004206</i>	
Gas Lot Number <i>28423080A3</i>	Gas Cylinder Number <i>4</i>	Gas Expiration Date <i>11/05/2025</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 - A. 0.083 AC
 - B. 0.083 AC
 - C. 0.083 AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

Chemical Test Operator Signature <i>[Signature]</i>	Date <i>12/12/24</i>
Reviewed By (Crime Laboratory Use Only) Anna Narehood <i>[Signature]</i>	Date 13Dec2024

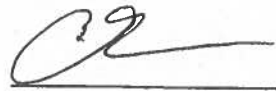
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004206
Location = CASS 8164.14.00 09/16
12/12/2024 23:21

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	23:22
02 Std. Gas	0.083	23:22
03 Room Air	0.000	23:23
04 Std. Gas	0.083	23:23
05 Room Air	0.000	23:24
06 Std. Gas	0.083	23:24
07 Room Air	0.000	23:25

Lot No = 28423080A3
Cyl No = 4
Exp Date = 11/05/2025
County = 09 Oper No. = 050910

 3691
Operator Signature
CHAD THOMPSON

Remarks: *New Cylinder Install*

Form 106-I8000