



# INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

NORTH DAKOTA OFFICE OF ATTORNEY GENERAL

CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM

SFN 59281 (06/2018)

Serial Number	20-04947	Instrument Location	1201 17th St SE, Suite C, Watford City, ND 58079
Reason for Install/Repair			
<input type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input checked="" type="checkbox"/> Other (Specify) <u>file uploaded at Crime Lab</u>			

### Check When Done:

- 1. Surge Protector Installed/Property Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
  - A. Print Test (Level 1, Function P).
  - B. ACA Test (Level 1, Function C).
  - C. Radio Frequency Interference (RFI) Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if any): N/A
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN504096, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature	<i>[Signature]</i>	Date	12/30/24
---------------------------	--------------------	------	----------

### Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified By	Anna Narehood	<i>[Signature]</i>	Certified Date	31Dec2024
-----------------------	---------------	--------------------	----------------	-----------

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-004947  
Location = MCSO                    8164.14.00 09/16  
12/30/2024                            16:14

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890- = |  
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%^&\* () \_ + ?

abcdefghijklmnopqrstuvwxyz1234567890- = |  
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%^&\* () \_ + ?

Current Instrument Setup


Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        Yes  
# of Print Copies?          1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            14323080A4  
Standard Cyl #?            15  
Standard Expiration?       06/05/2025  
Oper No?                    132007

Flow Cal. Date:            04/07/2020  
Slope                        618  
Intercept                   -565524

IR Calibration Date:       04/09/2024  
                                  3um                            9um

-----  
0th Coef(\*100):            -21002                        -23184  
1st Coef(\*100):            264237                        133599  
2nd Coef(\*100):            1927                            1247  
H2O adj(mg/l\*10k):        409                            438

\*\*\*\*\* Printer Test End \*\*\*\*\*

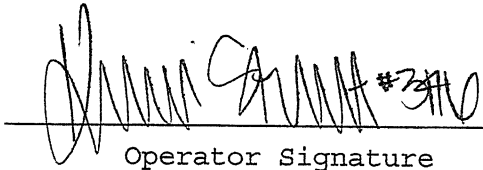

Operator Signature  
KARI STUART


Remarks:

STATE OF NORTH DAKOTA }  
COUNTY OF MCKENZIE }<sup>SS</sup>

I certify that the foregoing is a true and correct  
copy of the original document on file in this office.  
Dated at Watford City, North Dakota.

Form 106-I8000

on this 30th day of December, 2024

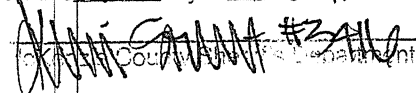

County Clerk

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004947  
Location = MCSO      8164.14.00 09/16  
12/30/2024      16:36

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:37
02 Std. Gas	0.079	16:37
03 Room Air	0.000	16:38
04 Std. Gas	0.080	16:38
05 Room Air	0.000	16:39
06 Std. Gas	0.080	16:39
07 Room Air	0.000	16:40

Lot No = 14323080A4  
Cyl No = 15  
Exp Date = 06/05/2025  
County = 27      Oper No. = 132007

*Kari Stuart #3410*

Operator Signature  
KARI STUART

Remarks:

*ACA*

Form 106-I8000

STATE OF NORTH DAKOTA }  
COUNTY OF MCKENZIE } ss

I certify that the foregoing is a true and correct  
copy of the original document on file in this office.  
Dated at Watford City, North Dakota.

on this 20 day of December 2024

*Kari Stuart #3410*  
MCKENZIE County Sheriff's Department

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004947  
Location = MCSO      8164.14.00 09/16  
12/30/2024      16:40

Test	AC	Time
01 Room Air	RFI*	16:41
02 Room Air	RFI*	16:41

\*Invalid Test  
Inhibited - RFI

Sub Name = TEST, TEST TEST  
Sub DOB = 01/01/1990  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = 00000000000  
Dr. Lic. = ND/000000000  
Lot No = 14323080A4  
Cyl No = 15

Expiration Date = 06/05/2025  
County = 27      Oper No. = 132007

*[Handwritten Signature]* #3410

Operator Signature  
KARI STUART

Remarks:

*[Handwritten: RFI]*

Form 106-I8000

STATE OF NORTH DAKOTA }  
COUNTY OF MCKENZIE } SS

I certify that the foregoing is a true and correct  
copy of the original document on file in this office.  
Dated at Watford City, North Dakota.

on this 20 day of December, 2024  
*[Handwritten Signature]* #3410  
McKenzie County Sheriff's Department