



ETHANOL GAS STANDARD CYLINDER REPORT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <i>Taylor Wray</i>		
Location <i>BMDL</i>	Intoxilyzer® Serial Number <i>80-004936</i>	
Gas Lot Number <i>01524080A4</i>	Gas Cylinder Number <i>27</i>	Gas Expiration Date <i>03/05/2026</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 - A. 0.079 AC
 - B. 0.080 AC
 - C. 0.081 AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

Chemical Test Operator Signature <i>Taylor Wray</i>	Date <i>01/15/2025</i>
Reviewed By (Crime Laboratory Use Only) <i>Ann Miller</i>	Date <i>16 Jan 2025</i>

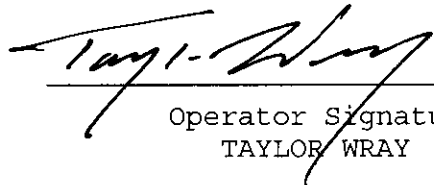
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004936
Location = BMDC 8164.14.00 09/16
01/15/2025 14:17

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:17
02 Std. Gas	0.079	14:18
03 Room Air	0.000	14:18
04 Std. Gas	0.080	14:19
05 Room Air	0.000	14:19
06 Std. Gas	0.081	14:19
07 Room Air	0.000	14:20

Lot No = 01524080A4
Cyl No = 27
Exp Date = 03/05/2026
County = 08 Oper No. = 131856


Operator Signature
TAYLOR WRAY

Remarks:

Form 106-I8000