



ETHANOL GAS STANDARD CYLINDER REPORT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <i>Tom Sharp</i>		
Location <i>New Salem</i>		Intoxilyzer® Serial Number <i>80-03059</i>
Gas Lot Number <i>015 24080A4</i>	Gas Cylinder Number <i>033</i>	Gas Expiration Date <i>3/5/26</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- AEN 2. Perform an ACA Test (Level 1, Function C).
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 - A. *0.081* AC
 - B. *0081* AC
 - C. *00.81* AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

Chemical Test Operator Signature 	Date <i>1/16/25</i>
Reviewed By (Crime Laboratory Use Only) Anna Narehood	Date 17Jan2025


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003059
Location = NS 8164.14.00 09/16
01/16/2025 15:45

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:46
02 Std. Gas	0.081	15:46
03 Room Air	0.000	15:46
04 Std. Gas	0.081	15:47
05 Room Air	0.000	15:47
06 Std. Gas	0.081	15:48
07 Room Air	0.000	15:48

Lot No = 01524080A4
Cyl No = 33
Exp Date = 03/05/2026
County = 30 Oper No. = 020611

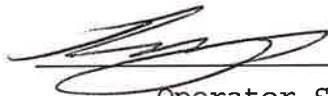

Operator Signature
TOM SHARP

Remarks:

Form 106-I8000