



ETHANOL GAS STANDARD CYLINDER REPORT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59282 (08/2018)

Chemical Test Operator Name (Print) FRANK A SENN		
Location BEULAH POLICE DEPARTMENT	Intoxilyzer® Serial Number 80-003058	
Gas Lot Number 01524080A4	Gas Cylinder Number 34	Gas Expiration Date 03-05-2026

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 A. 0.081 AC
 B. 0.081 AC
 C. 0.082 AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

Chemical Test Operator Signature 	Date 2-6-2025
Reviewed By (Crime Laboratory Use Only) Anna Narehood	Date 07Feb2025

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003058
Location = BEPD 8164.14.00 09/16
02/06/2025 16:43

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:44
02 Std. Gas	0.081	16:44
03 Room Air	0.000	16:44
04 Std. Gas	0.081	16:45
05 Room Air	0.000	16:45
06 Std. Gas	0.082	16:46
07 Room Air	0.000	16:46

Lot No = 01524080A4
Cyl No = 34
Exp Date = 03/05/2026
County = 29 Oper No. = 039714



Operator Signature
FRANK SENN

Remarks:

Form 106-I8000