

INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

NORTH DAKOTA OFFICE OF ATTORNEY GENERAL CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM SFN 59281 (06/2018)

Serial Number	Instrument Location			
80 ~ 00 3065 Reason for Install/Repair	RICHLAND COUNTY JAZL			
🕅 Install After Receiving From Crime Laboratory 🛛 Install After Location Change				
Other (Specify)				
Check When Done:				
X 1. Surge Protector Installed/Property Grounded.				
2. Telephone Line Connected to Intoxilyzer® 8000.				
X 3. Breath Tube Heated.				
🔀 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).				
🔀 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).				
🔀 6. Run Tests:				
X A. Print Test (Level1, Function P).				
B. ACA Test (Level 1, Function C).				
🕅 C. Radio Frequency Interference (RFI) Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).				
7. Repair and/or Maintenance Performed (if any):				
8. Complete the Top Portion of the Intoxilyzer® Record (SFN5	i0496, Form 120-G) and Place it by the Intoxilyzer® for Use.			
9. File Previous Intoxilyzer® Record (SFN504096, Form 120-G) at the Intoxilyzer® Location at the Agency.				
104-G), Print Test, ACA Test and BFI Test.	toxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form			
Field Inspector Signature	Date 02/26/25			
Crime Laboratory Use Only				
This installation has been reviewed and the instrument is approved to be the date the Field Inspector performed the installation. This record on the County of Burleigh, North Dakota, is certified to be a true and correct co				

Reviewed/Certified By Anna Narehood	Am	Mall	Certified Date 14March2025	
	-0.0.000			

FORM 104-G

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501 CMI, Inc. Intoxilyzer Alcohol Analyzer North Dakota Model 8000 SN 80-003065 Location = WAHP 8164.14.00 09/16 13:08 02/26/2025 abcdefghijklmnopqrstuvwxyz1234567890-= ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*() +? abcdefghijklmnopgrstuvwxyz1234567890-=| ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*() +? Current Instrument Setup Data Entry Mode: Enabled Start Test Sequence: DABACABA Display Prelim Rslt? Yes Display Third Digit? Yes Inhib Printer(Y/N)? NO Display Volume? Νo Disable On Memfull? Yes # of Print Copies? 1 Select Std (D/W/I)? Dry Standard Value? 0.080 Standard Lot #? 28423080A3 Standard Cyl #? 47 Standard Expiration? 11/05/2025 Oper No? 132570 Flow Cal. Date: 05/19/2017 612 Slope 21223 Intercept IR Calibration Date: 08/23/2016 3um 9um _ _ _ _ _ _ _ _ _ _ -12309 0th Coef(*100): -14934 1st Coef(*100): 289519 141483 2nd Coef(*100): 1834 736 136 H2O adj(mg/l*10k): 268 Operator Signature CHRISTOPHER HUARD

Remarks: GOOD PRINT

Form 106-18000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. IntoxilyzerAlcohol AnalyzerNorth Dakota Model 8000SN 80-003065Location = WAHP8164.14.00 09/1602/26/202513:08

DRY CAL CHECK				
Test	AC	Time		
01 Room Air 02 Std. Gas 03 Room Air 04 Std. Gas 05 Room Air 06 Std. Gas 07 Room Air	0.000 0.080 0.000 0.078 0.000 0.078 0.000	13:09 13:10 13:10 13:10 13:11 13:11 13:11 13:12		

Lot No = 28423080A3 Cyl No = 47 Exp Date = 11/05/2025 County = 39

Oper No. = 132570

Operator Signature CHRISTOPHER HUARD

Remarks: 6000

ACA

Form 106-18000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501 Alcohol Analyzer CMI, Inc. Intoxilyzer SN 80-003065 North Dakota Model 8000 Location = WAHP 8164.14.00 09/16 13:12 02/26/2025 Test AC Time 01 Room Air RFI* 13:14 0.000 13:14 02 Room Air *Invalid Test Inhibited - RFI Sub Name = SUBJECT, TEST NA Sub DOB = 01/01/1987Sub Sex = Male Weight = 210Cit = 12345678 Test = OTHDr. Lic. = ND/12345678Lot No = 28423080A3Cyl No = 47Expiration Date = 11/05/2025Oper No. = 132570County = 39

Operator Signature CHRISTOPHER HUARD

Remarks:

6000

. .

RFZ

Form 106-18000