

**INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT**

NORTH DAKOTA OFFICE OF ATTORNEY GENERAL

CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM

SFN 59281 (06/2018)

Serial Number	<i>SD-006490</i>	Instrument Location	<i>MTJ 1053</i>
Reason for Install/Repair			
<input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change			
<input type="checkbox"/> Other (Specify) _____			

## Check When Done:

- ☒ 1. Surge Protector Installed/Property Grounded.
- ☒ 2. Telephone Line Connected to Intoxilyzer® 8000.
- ☒ 3. Breath Tube Heated.
- ☒ 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- ☒ 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- AEN ☒ 6. Run Tests:
- ☒ A. Print Test (Level 1, Function P).
- ☒ B. ACA Test (Level 1, Function C).
- ☒ C. Radio Frequency Interference (RFI) Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- ☒ 7. Repair and/or Maintenance Performed (if any): \_\_\_\_\_
- ☒ 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- ☒ 9. File Previous Intoxilyzer® Record (SFN50496) Form 120-G at the Intoxilyzer® Location at the Agency.
- ☒ 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature	<i>[Signature]</i>	Date	<i>4/2/25</i>
---------------------------	--------------------	------	---------------

## Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified By	<i>Anna Narehood</i>	Certified Date	4/14/2025
-----------------------	----------------------	----------------	-----------

CMI, Inc. Intoxilyzer	Alcohol Analyzer
North Dakota Model 8000	SN 80-006490
Location = ASHL	8164.14.00 09/16
04/02/2025	17:13

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006490  
Location = ASHL      8164.14.00 09/16  
04/02/2025      17:13

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	17:14
02 Std. Gas	0.081	17:14
03 Room Air	0.000	17:15
04 Std. Gas	0.080	17:15
05 Room Air	0.000	17:16
06 Std. Gas	0.080	17:16
07 Room Air	0.000	17:17


Lot No = 01524080A4

Cyl No = 20

Exp Date = 03/05/2026

County = 26

Oper No. = 130117

  
Operator Signature  
MARK PEARSON

Remarks:

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006490  
Location = ASHL      8164.14.00 09/16  
04/02/2025      17:17

Test	AC	Time
01 Room Air	RFI*	17:19
02 Room Air	0.000	17:19

\*Invalid Test  
Inhibited - RFI

Sub Name = PEARSON, MARK ALAN  
Sub DOB = 02/28/1962  
Sub Sex = Male      Weight = 250  
Test = OTH      Cit = NA  
Dr. Lic. = ND/PEA629285  
Lot No = 01524080A4  
Cyl No = 20  
Expiration Date = 03/05/2026  
County = 15      Oper No. = 130117

  
Operator Signature  
MARK PEARSON

Remarks:

Form 106-I8000