



# ETHANOL GAS STANDARD CYLINDER REPORT

NORTH DAKOTA OFFICE OF ATTORNEY GENERAL

CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM

SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <i>Pamela B. Pehl</i>		
Location <i>Stat</i>	Intoxilyzer® Serial Number <i>80-005944</i>	
Gas Lot Number <i>25024080 A3</i>	Gas Cylinder Number <i>59</i>	Gas Expiration Date <i>10-05-2026</i>

Check When Done:

- ☒ 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- ☒ 2. Perform an ACA Test (Level 1, Function C).  
Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
  - A. 0. *080* AC
  - B. 0. *080* AC
  - C. 0. *080* AC
- ☒ 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- ☒ 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- ☒ 5. Send the Following to the Crime Laboratory:
  - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
  - B. ACA Test Record.

Chemical Test Operator Signature <i>P. Pehl</i>	Date <i>4/14/2025</i>
Reviewed By (Crime Laboratory Use Only) <i>Anna Narehood</i>	Date <i>16 April 2025</i>

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005944  
Location = STAT      8164.14.00 09/16  
04/14/2025      12:57

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:57
02 Std. Gas	0.080	12:58
03 Room Air	0.000	12:58
04 Std. Gas	0.081	12:59
05 Room Air	0.000	12:59
06 Std. Gas	0.080	13:00
07 Room Air	0.000	13:00


Lot No = 25024080A3

Cyl No = 59

Exp Date = 10/05/2026

County = 29

Oper No. = 120906

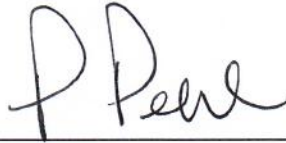

Operator Signature  
PAMELA PEHL

Remarks:

Form 106-I8000