

INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

NORTH DAKOTA OFFICE OF ATTORNEY GENERAL CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM SFN 59281 (06/2018)

Serial Number	Instrument Location	-	
80-004942 Fargo Police Department Reason for Install/Repair			
The contraction of the contracti			
	er Location Change		
Other (Specify) Reinstallation following 202	5 Annual Inspection at Crime	Laboratory).	
Check When Done:			
2. Telephone Line Connected to Intoxilyzer® 8000.			
☑ 3. Breath Tube Heated.			
4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Locati	4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).		
≤ 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).	5. Scan/Enter Gas Cylinder Information (Level 1, Function S).		
∑ 6. Run Tests:			
☑ C. Radio Frequency Interference (RFI) Test (CMS Mode or CMS)	Level 1, Function B or C; Key Radio During	g Test).	
⊼ Repair and/or Maintenance Performed (if any): No No.	laintenance Performed		
8. Complete the Top Portion of the Intoxilyzer® Record (SFN5	0496, Form 120-G) and Place it by the Into	kilyzer® for Use.	
☑ 9. File Previous Intoxilyzer® Record (SFN504096, Form 120-0)	G) at the Intoxilyzer® Location at the Agency	/.	
10. Send the Following to the Crime Laboratory: Completed Inf 104-G), Print Test, ACA Test, and RFI Test.	toxilyzer® 8000 Installation and Repair Ched	ckout (SFN59281, Form	
Field Inspector Signature	Da	ate D4/14/2025	
Crime Laboratory Use Only			
This installation has been reviewed and the instrument is approved to be the date the Field Inspector performed the installation. This record on the County of Burleigh, North Dakota, is certified to be a true and correct county.	file at the Office of Attorney General, Crime		
Reviewed/Certified By Anna Narehood	M 1 () C	ertified Date	

FORM 104-G

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer	Alcohol Analyzer
North Dakota Model 8000	SN 80-004942
Location = FAPD	8164.14.00 09/16
04/14/2025	16:31

******* Printer Test ********

abcdefghijklmnopqrstuvwxyz1234567890-=|
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=|
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*() +?

Flow Cal.	Date:	05/25/2018
Slope		626
Intercept		-548004

IR Calibration Date: 04/09/2024

	3um	9um
Coef(*100): Coef(*100):	 -22609 260445	 -23575 132718
Coef(*100): adj(mg/l*10k):	2885 492	1338 488

****** Printer Test End *******

Operator Signature PATRICK THOMAS

Remarks: Reinstallation after annual inspection

Form 106-I8000

NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer
North Dakota Model 8000
Location = FAPD
04/14/2025

Alcohol Analyzer SN 80-004942 8164.14.00 09/16 16:32

DRY CAL CHECK

T e	est		AC	Time
01	Room	Air	0.000	16:33
02	Std.	Gas	0.082	16:33
03	Room	Air	0.000	16:33
04	Std.	Gas	0.081	16:34
05	Room	Air	0.000	16:34
06	Std.	Gas	0.081	16:35
07	Room	Air	0.000	16:35

Lot No = 01524080A4

Cyl No = 32

Exp Date = 03/05/2026

County = 09

Oper No. = 130056

Operator Signature PATRICK THOMAS

Remarks: Reinstallation after annual inspection

Form 106-I8000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer North Dakota Model 8000 SN 80-004942 Location = FAPD 8164.14.00 09/16 04/14/2025 16:40

Test			AC	Time
01	Room	Air	RFI*	16:42
02	Room	Air	0.000	16:43

*Invalid Test Inhibited - RFI

Sub Name = REINSTALLATION, ANNUAL INSPECTION

Sub DOB = 01/01/2000

Sub Sex = Male

Test = DUI

Weight = 150Cit = NA

Dr. Lic. = ND/NALot No = 01524080A4

Cyl No = 32

Expiration Date = 03/05/2026

County = 09

Oper No. = 130056

Operator Signature PATRICK THOMAS

Remarks: Reinstallation after annual inspection

Form 106-I8000