



# ETHANOL GAS STANDARD CYLINDER REPORT

NORTH DAKOTA OFFICE OF ATTORNEY GENERAL

CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM

SFN 59282 (08/2018)

|                                                              |                                  |                                                |
|--------------------------------------------------------------|----------------------------------|------------------------------------------------|
| Chemical Test Operator Name (Print)<br><b>MICHAEL MAROLT</b> |                                  |                                                |
| Location<br><b>N APO</b>                                     |                                  | Intoxilyzer® Serial Number<br><b>80-004200</b> |
| Gas Lot Number<br><b>2502408043</b>                          | Gas Cylinder Number<br><b>60</b> | Gas Expiration Date<br><b>10-05-2026</b>       |

## Check When Done:

- ☒ 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- ☒ 2. Perform an ACA Test (Level 1, Function C).  
Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
  - A. 0.002 AC
  - B. 0.002 AC
  - C. 0.002 AC
- ☒ 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- ☒ 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- ☒ 5. Send the Following to the Crime Laboratory:
  - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
  - B. ACA Test Record.

|                                                                     |                              |
|---------------------------------------------------------------------|------------------------------|
| Chemical Test Operator Signature<br>                                | Date<br><b>4-17-2025</b>     |
| Reviewed By (Crime Laboratory Use Only)<br><b>Anna Narehood</b><br> | Date<br><b>23 April 2025</b> |


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004200  
Location = NAPO      8164.14.00 09/16  
04/17/2025      14:42

DRY CAL CHECK

| Test        | AC    | Time  |
|-------------|-------|-------|
| 01 Room Air | 0.000 | 14:43 |
| 02 Std. Gas | 0.082 | 14:43 |
| 03 Room Air | 0.000 | 14:43 |
| 04 Std. Gas | 0.082 | 14:44 |
| 05 Room Air | 0.000 | 14:44 |
| 06 Std. Gas | 0.082 | 14:45 |
| 07 Room Air | 0.000 | 14:45 |

Lot No = 25024080A3  
Cyl No = 60  
Exp Date = 10/05/2026  
County = 24      Oper No. = 131100

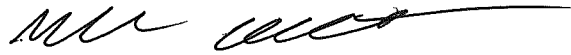

Operator Signature  
MICHAEL MAROLT

Remarks:

Form 106-I8000