



# ETHANOL GAS STANDARD CYLINDER REPORT

NORTH DAKOTA OFFICE OF ATTORNEY GENERAL

CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM

SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <i>Taylor Wray</i>		
Location <i>BMDL</i>	Intoxilyzer® Serial Number <i>80-005956</i>	
Gas Lot Number <i>28423080A3</i>	Gas Cylinder Number <i>06</i>	Gas Expiration Date <i>11/05/2025</i>

## Check When Done:

- ☒ 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- ☒ 2. Perform an ACA Test (Level 1, Function C).  
Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
  - A. *0.078* AC
  - B. *0.078* AC
  - C. *0.079* AC
- ☒ 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- ☒ 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- ☒ 5. Send the Following to the Crime Laboratory:
  - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
  - B. ACA Test Record.

Chemical Test Operator Signature <i>Taylor Wray</i>	Date <i>04/21/2025</i>
Reviewed By (Crime Laboratory Use Only) <i>Anna Narehood</i>	Date <i>25 April 2025</i>

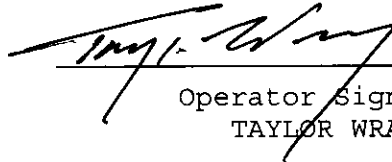
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005956  
Location = BMDC      8164.14.00 09/16  
04/21/2025      11:32

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:32
02 Std. Gas	0.078	11:33
03 Room Air	0.000	11:33
04 Std. Gas	0.078	11:34
05 Room Air	0.000	11:34
06 Std. Gas	0.079	11:34
07 Room Air	0.000	11:35

Lot No = 28423080A3  
Cyl No = 06  
Exp Date = 11/05/2025  
County = 08      Oper No. = 131856

  
Operator Signature  
TAYLOR WRAY

Remarks:

Form 106-I8000