



ETHANOL GAS STANDARD CYLINDER REPORT

NORTH DAKOTA OFFICE OF ATTORNEY GENERAL

CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM

SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <i>Gabriel Johnston</i>		
Location <i>Langdon</i>		Intoxilyzer® Serial Number <i>80-003061</i>
Gas Lot Number <i>2502408013</i>	Gas Cylinder Number <i>8</i>	Gas Expiration Date <i>10-05-2026</i>

Check When Done:

☒ 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).

☒ 2. Perform an ACA Test (Level 1, Function C).

Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).

A. 0. *080* AC

B. 0. *081* AC

C. 0. *082* AC

☒ 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.

☒ 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.

☒ 5. Send the Following to the Crime Laboratory:

A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).

B. ACA Test Record.

Chemical Test Operator Signature <i>[Signature]</i>	Date <i>4-24-2025</i>
Reviewed By (Crime Laboratory Use Only) <i>Anna Narehood</i> <i>Ann Muel</i>	Date <i>30 April 2025</i>

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = LANG 8164.14.00 09/16
04/24/2025 08:35

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	08:36
02 Std. Gas	0.080	08:36
03 Room Air	0.000	08:36
04 Std. Gas	0.081	08:37
05 Room Air	0.000	08:37
06 Std. Gas	0.080	08:38
07 Room Air	0.000	08:38


Lot No = 25024080A3

Cyl No = 008

Exp Date = 10/05/2026

County = 10

Oper No. = 021009

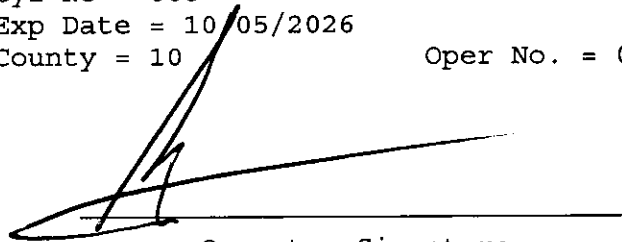

Operator Signature
GABRIEL JOHNSTON

Remarks:

Form 106-I8000