



ETHANOL GAS STANDARD CYLINDER REPORT

NORTH DAKOTA OFFICE OF ATTORNEY GENERAL

CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM

SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <i>Grossbauer, John</i>		
Location <i>GFTN</i>	Intoxilyzer® Serial Number <i>80-004945</i>	
Gas Lot Number <i>01524080A4</i>	Gas Cylinder Number <i>31</i>	Gas Expiration Date <i>3-5-2026</i>

Check When Done:

☒ 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).

☒ 2. Perform an ACA Test (Level 1, Function C).

Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).

A. *0.078* AC

B. *0.078* AC

C. *0.079* AC

☒ 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.

☒ 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.

☒ 5. Send the Following to the Crime Laboratory:

A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).

B. ACA Test Record.

Chemical Test Operator Signature <i>[Signature]</i>	Date <i>4-28-2025</i>
Reviewed By (Crime Laboratory Use Only) Anna Narehood <i>[Signature]</i>	Date 29April2025

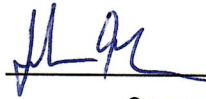
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004945
Location = GFTN 8164.14.00 09/16
04/28/2025 07:31

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	07:31
02 Std. Gas	0.078	07:32
03 Room Air	0.000	07:32
04 Std. Gas	0.078	07:33
05 Room Air	0.000	07:33
06 Std. Gas	0.079	07:33
07 Room Air	0.000	07:34

Lot No = 01524080A4
Cyl No = 31
Exp Date = 03/05/2026
County = 50 Oper No. = 133368

 #1441

Operator Signature
JOHN GROSSBAUER

Remarks:

Form 106-I8000