

**INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT**

NORTH DAKOTA OFFICE OF ATTORNEY GENERAL

CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM

SFN 59281 (06/2018)

Serial Number 80-007096	Instrument Location DEVL
Reason for Install/Repair ☒ Install After Receiving From Crime Laboratory ☐ Install After Location Change ☐ Other (Specify) _____	

Check When Done:

- ☒ 1. Surge Protector Installed/Property Grounded.
- ☒ 2. Telephone Line Connected to Intoxilyzer® 8000.
- ☒ 3. Breath Tube Heated.
- ☒ 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- ☒ 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- ☒ 6. Run Tests:
 - ☒ A. Print Test (Level 1, Function P).
 - ☒ B. ACA Test (Level 1, Function C).
 - ☒ C. Radio Frequency Interference (RFI) Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- ☒ 7. Repair and/or Maintenance Performed (if any): N/A
- ☒ 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- ☒ 9. File Previous Intoxilyzer® Record (SFN504096, Form 120-G) at the Intoxilyzer® Location at the Agency.
- ☒ 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature 	Date 5/7/2025
-------------------------------	-------------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified By Anna Narehood	Certified Date 08May2025
---	------------------------------------

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007096
Location = DEVL 8164.16.00 09/18
05/07/2025 10:28

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-=
ABCDEFGHIJKLMN0PQRSTUVWXYZ!@#%^&*()_+?

abcdefghijklmnopqrstuvwxy1234567890-=
ABCDEFGHIJKLMN0PQRSTUVWXYZ!@#%^&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 08425080A1
Standard Cyl #? 018
Standard Expiration? 05/05/2027
Oper No? 132008

Flow Cal. Date: 10/02/2018
Slope 695
Intercept -692605

IR Calibration Date: 10/02/2018
 3um 9um

0th Coef(*100):	-33544	-18475
1st Coef(*100):	261464	138054
2nd Coef(*100):	3020	1612
H2O adj(mg/l*10k):	573	386

***** Printer Test End *****

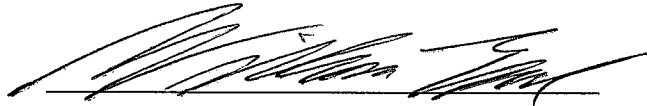

Operator Signature
WILLIAM EARL

Remarks:

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007096
Location = DEVL 8164.16.00 09/18
05/07/2025 10:24

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:24
02 Std. Gas	0.080	10:25
03 Room Air	0.000	10:25
04 Std. Gas	0.079	10:26
05 Room Air	0.000	10:26
06 Std. Gas	0.079	10:27
07 Room Air	0.000	10:27

Lot No = 08425080A1
Cyl No = 018
Exp Date = 05/05/2027
County = 36 Oper No. = 132008

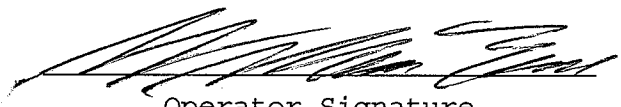
Operator Signature
WILLIAM EARL

Remarks:

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007096
Location = DEVL 8164.16.00 09/18
05/07/2025 10:29

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:30
02 Std. Gas	0.079	10:30
03 Room Air	RFI*	10:31
04 Room Air	0.000	10:31

*Invalid Test
Inhibited - RFI

Lot No = 08425080A1
Cyl No = 018
Exp Date = 05/05/2027
County = 36 Oper No. = 132008



Operator Signature
WILLIAM EARL

Remarks:

Form 106-I8000