



**INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT**  
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM  
 SFN 59281 (06/2018)

Serial Number <b>81-004947</b>	Instrument Location <b>DCSO</b>
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Property Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
  - A. Print Test (Level1, Function P).
  - B. ACA Test (Level 1, Function C).
  - C. Radio Frequency Interference (RFI) Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if any): \_\_\_\_\_
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN504096, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature 	Date <b>5/7/25</b>
-------------------------------	-----------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified By <b>Anna Narehood</b>	Certified Date <b>07May2025</b>
---	------------------------------------

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004947  
Location = DUNN      8164.14.00 09/16  
05/07/2025      17:50

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxy1234567890-|=|  
ABCDEFGHIJKLMNopQRSTUVWXYZ!@#\$\$%^&\*()\_+?

abcdefghijklmnopqrstuvwxy1234567890-|=|  
ABCDEFGHIJKLMNopQRSTUVWXYZ!@#\$\$%^&\*()\_+?

Current Instrument Setup

Data Entry Mode:      Enabled  
Start Test Sequence:      DABACABA  
Display Prelim Rslt?      Yes  
Display Third Digit?      Yes  
Inhib Printer(Y/N)?      No  
Display Volume?      No  
Disable On Memfull?      Yes  
# of Print Copies?      1  
Select Std (D/W/I)?      Dry  
Standard Value?      0.080  
Standard Lot #?      01524080A4  
Standard Cyl #?      21  
Standard Expiration?      03/05/2026  
Oper No?      132823

Flow Cal. Date:      04/07/2020  
Slope      618  
Intercept      -565524

IR Calibration Date:      04/09/2024

	3um	9um
0th Coef(*100):	-21002	-23184
1st Coef(*100):	264237	133599
2nd Coef(*100):	1927	1247
H2O adj(mg/l*10k):	409	438

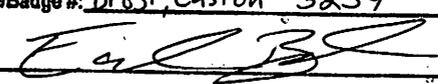
\*\*\*\*\* Printer Test End \*\*\*\*\*



Operator Signature  
EASTON BROST

Remarks:

Form 106-I8000

<b>I certify that the attached is a true and correct copy of the intoxilyzer test record.</b>	
Print Name/Badge #:	Brost, Easton 3254
Signature:	
Date:	5/7/25
DUNN COUNTY SHERIFF'S OFFICE	

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004947  
Location = DUNN      8164.14.00 09/16  
05/07/2025      17:51

DRY CAL CHECK

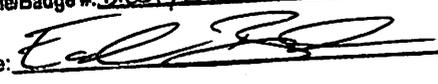
Test	AC	Time
01 Room Air	0.000	17:51
02 Std. Gas	0.078	17:52
03 Room Air	0.000	17:52
04 Std. Gas	0.078	17:53
05 Room Air	0.000	17:53
06 Std. Gas	0.078	17:53
07 Room Air	0.000	17:54

Lot No = 01524080A4  
Cyl No = 21  
Exp Date = 03/05/2026  
County = 13      Oper No. = 132823

  
Operator Signature  
EASTON BROST

Remarks:

Form 106-I8000

I certify that the attached is a true and correct copy of the intoxilyzer test record.  
Print Name/Badge #: Brost, Easton 3254  
Signature:   
Date: 5/7/25 DUNN COUNTY SHERIFF'S OFFICE

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004947  
Location = DUNN      8164.14.00 09/16  
05/07/2025      17:55

Test	AC	Time
01 Room Air	RFI*	17:57
02 Room Air	0.000	17:58

\*Invalid Test  
Inhibited - RFI

Sub Name = BROST, EASTON A  
Sub DOB = 09/19/1996  
Sub Sex = Male      Weight = 185  
Test = DUI      Cit = 40031290  
Dr. Lic. = ND/BRO968185  
Lot No = 01524080A4  
Cyl No = 21  
Expiration Date = 03/05/2026  
County = 13      Oper No. = 132823



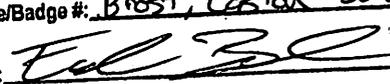
Operator Signature  
EASTON BROST

Remarks:

Form 106-I8000

I certify that the attached is a true and correct copy of the intoxilyzer test record.

Print Name/Badge #: Brost, Easton 3254

Signature: 

Date: 5/7/25 DUNN COUNTY SHERIFF'S OFFICE