



ETHANOL GAS STANDARD CYLINDER REPORT
NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <i>Jon Ward</i>		
Location <i>SARGENT county Sheriff's office</i>		Intoxilyzer® Serial Number <i>80-006690</i>
Gas Lot Number <i>08425080A1</i>	Gas Cylinder Number <i>11</i>	Gas Expiration Date <i>05/08/27</i>

Check When Done:

- ☒ 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- ☒ 2. Perform an ACA Test (Level 1, Function C).
Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 - A. *0.080* AC
 - B. *0.080* AC
 - C. *0.080* AC
- ☒ 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- ☒ 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- ☒ 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

Chemical Test Operator Signature <i>Jon Ward</i>	Date <i>5/27/25</i>
Reviewed By (Crime Laboratory Use Only) Anna Narehood	Date 30May2025

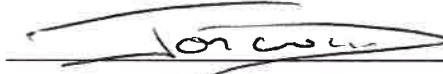
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006690
Location = SACO 8164.14.00 09/16
05/27/2025 18:29

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	18:30
02 Std. Gas	0.080	18:30
03 Room Air	0.000	18:31
04 Std. Gas	0.080	18:31
05 Room Air	0.000	18:32
06 Std. Gas	0.080	18:32
07 Room Air	0.000	18:33

Lot No = 08425080A1
Cyl No = 11
Exp Date = 05/05/2027
County = 41 Oper No. = 132607


Operator Signature
JONATHAN WOLD

Remarks:

Form 106-I8000