



ETHANOL GAS STANDARD CYLINDER REPORT

NORTH DAKOTA OFFICE OF ATTORNEY GENERAL

CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM

SFN 59282 (08/2018)

| | | |
|---|----------------------------------|--|
| Chemical Test Operator Name (Print) <i>Tim Bleth</i> | | |
| Location <i>BSP0</i> | | Intoxilyzer® Serial Number <i>80-005945</i> |
| Gas Lot Number <i>08425080A1</i> | Gas Cylinder Number <i>21</i> | Gas Expiration Date <i>05/05/27</i> |

Check When Done:

- ☒ 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- AEN ☒ 2. Perform an ACA Test (Level 1, Function C).
06/10/25 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
A. 0.079 AC
B. 0.079 AC
C. 0.079 AC
- ☒ 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- ☒ 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- ☒ 5. Send the Following to the Crime Laboratory:
A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
B. ACA Test Record.

| | |
|--|-----------------------|
| Chemical Test Operator Signature <i>T. Bleth</i> | Date <i>6-5-25</i> |
| Reviewed By (Crime Laboratory Use Only) Anna Narehood <i>Anna Narehood</i> | Date 10June2025 |


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005945
Location = BSPD 8164.14.00 09/16
06/05/2025 14:24

DRY CAL CHECK

| Test | AC | Time |
|-------------|-------|-------|
| 01 Room Air | 0.000 | 14:24 |
| 02 Std. Gas | 0.079 | 14:25 |
| 03 Room Air | 0.000 | 14:25 |
| 04 Std. Gas | 0.079 | 14:26 |
| 05 Room Air | 0.000 | 14:26 |
| 06 Std. Gas | 0.079 | 14:27 |
| 07 Room Air | 0.000 | 14:27 |

Lot No = 08425080A1
Cyl No = 21
Exp Date = 05/05/2027
County = 08 Oper No. = 069823


Operator Signature
TIMOTHY BLETH

Remarks: *New Cylinder ACA*

Form 106-I8000