

**NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION**

INTOXILYZER® 8000 INITIAL INSPECTION

Intoxilyzer® 8000 Serial Number: 80-00 5951

A. General Setup and Checks:

1. Initial start up
 - ☒ Power On
 - ☒ 5-minute countdown during initial warm-up
 - ☒ 4-minute countdown to begin test after standby mode
2. ☒ Display Intensity (Ctrl + ↑ or ↓)
3. Password Security:
 - ☒ Level 1 (Operator)
 - ☒ Level 2 (Field Inspector)
 - ☒ Level 3 (Lab Personnel)
4. ☒ Printer Setup (Level 3,S,P)
5. ☒ Forms Load Completed (CMI, Inc., CoBrA Software)

B. Level 1 Setup and Checks:

1. ☒ Internal Printer Test (Level 1,P). Sign and attach record.
 - ☒ Paper Feed
2. ☒ External Printer Test (Level 1,P). Sign and attach record.
3. Barcode Reader (Level 1,B) (Tone sounds after each)
 - ☒ Operator ID
 - ☒ Invalid ID
 - ☒ ND License
 - ☒ R key: Refusal. Sign and attach record.

C. Level 2 Setup and Checks:

1. ☒ Continuous Air Blank (Level 2,A)
2. ☒ Diagnostic (Level 2,D). Sign and attach test record.
3. ☒ Prelim Data Entry (Level 2,E)
4. Ser Number/Version Info (Level 2,V):
 - Inst Ser#: 80-005951 DSP Ver: 1362.14
 - Core Ver: 8164.14 CPLD Ver: 7

D. Level 3 Setup and Checks:

1. DVM Monitor (Level 3,D,D):

3: 12448 s: 2 C: 47.00

9: 12835 s: 2 B: 45

☒ DVM (3 & 9) between 9,000-16,000 counts

☒ RFI indicator with keyed radio: 0 → 1

2. Function (F) Keys:

☒ F5 – Tone

☒ F9 - Fan

☒ F6 – Green LED

☒ F10 – Pump

☒ F7 – Red LED

☒ F11 – Solenoid

3. Atmospheric Monitor (Level 3,D,A): Atmo Sensor, P: 954 CF: 1.06

4. Flow Monitor (Level 3,D,F): Flow Sensor, R: 228 F: 0.000 L/s

5. Temperature Monitor (Level 3,D,T):

C: 47.0 A: 48.5 I: 0.0

B: 45.0 V: 0.0

☒ Cell Temperature (C) between 47 ± 0.02 °C

☒ Breath Hose Temperature (B) between 35.5 – 47 °C

☒ Internal Ambient Temperature (A) above 41 °C

6. Tank Monitor (Level 3,D,G):

Display: 482 psi Regulator: 500 psi

Display and Regulator ± 50 psi of each other ☒ Yes or No

Gas tank tare necessary? Yes or ☒ No

If Yes, display readings after tare (Level 3,M,C,G):

Display: _____ psi Regulator: _____ psi

7. ☒ General Setup (Level 3,S,G)

8. ☒ Configure Start Test (Level 3,S,T)

9. ☒ Network (Level 3,S,N)

E. Tests (Sign and attach all test records):

1. ☒ Configure wet bath simulator for the following test (Level 1,S)

Zero AC-Wet Calibration Check (Level 1,C):

Sim ser #: MP3003 Lot #: NIA Exp date: NIA

Known value: 0.000 AC

☒ Results are Zero AC

2. ☒ Configure wet bath simulator for the following test (Level 1,S)

Low AC-Wet Calibration Check (Level 1,C):

Known Value ≤ 0.03 AC: 0.020 AC

Sim ser #: MP5321 Lot #: 202311D Exp date: 11/28/25

☒ Results ± 0.005 of known AC

3. ☒ Configure wet bath simulator for the following test (Level 1,S)
 High AC-Wet Calibration Check (Level 1,C):
 Known Value ≥ 0.25 AC: 0.300 AC
 Sim ser #: MP6035 Lot #: 202402C Exp date: 2/14/26
☒ Results $\pm 5\%$ of known AC
4. ☒ Configure dry gas standard for the following tests (Level 1,S)
 Known Value: 0.080 AC
 Gas Cylinder Lot #: 2842308043 Cylinder #: 41 Exp. Date: 11/5/25
5. 0.10% Ethanol + 0.05% Acetone (Level 1,B):
 Sim. Ser #: DR7352 Lot #: ICS 8 Exp. Date: None
☒ Display reads "Interferent Detect"
6. 0.05% Acetone (Level 1,B):
 Sim. Ser #: DR7111 Lot #: ACE 4 Exp. Date: None
☒ Display reads "Interferent Detect"
7. 0.10% Methanol (Level 1,B):
 Sim. Ser #: DR7345 Lot #: MET 4 Exp. Date: None
☒ Display Reads "Interferent Detect"
8. 0.10% Isopropanol (Level 1,B):
 Sim. Ser #: DR7350 Lot #: IPA 4 Exp. Date: None
☒ Display Reads "Interferent Detect"
9. ☒ Ambient Fail (Level 1,B): Display read and test record correct.
10. ☒ Invalid Sample (Level 1,B): Display read and test record correct.
11. ☒ Sequence Aborted (Level 1,B): Display read and test record correct.
12. ☒ ABA (Level 1,B): Test record correct.
13. ☒ Calibration check out of tolerance (CMS mode): Display read and test record correct.
14. ☒ RFI Detected (B,C, or CMS mode): Display read and test record correct.
15. ☒ Tank Pressure Below Minimum (CMS mode): Display reads correct.
16. ☒ Deficient Samples (CMS mode): Display read and test record correct.
17. ☒ Improper Sample (CMS mode): Display read and test record correct.
18. ☒ Difference Too Great (CMS mode): Display read and test record correct.
19. ☒ CMS (CMS mode): Test record correct.
20. Stability Test (Level 3,T,S):
 Ave: 0.0783 Std Dev: 0.0007 Rel Std Dev: 0.84%


Maintenance/Repairs/Notes: NIA

Instrument is acceptable to be used in the field.
If No, state reason(s) why:

☒ Yes or No


Inspector Signature

18 June 2025
Date

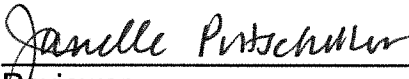
Reviewer

20 June 2025
Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005951
Location = TOXL 8164.14.00 09/16
06/18/2025 09:58

***** Printer Test *****

12 char/line
abcdefghijkl
ABCDEFGHIJKL

16 char per line
abcdefghijklmnop
ABCDEFGHIJKLMN

21 chars per line
abcdefghijklmnopqrstu
ABCDEFGHIJKLMNQRSTU

24 characters per line
abcdefghijklmnopqrstuvw
ABCDEFGHIJKLMNQRSTUW

32 characters per line
abcdefghijklmnopqrstuvwxy123456
ABCDEFGHIJKLMNQRSTUWXY123456


42 characters per line
abcdefghijklmnopqrstuvwxy1234567890abcdef
ABCDEFGHIJKLMNQRSTUWXY1234567890ABCDEF

Current Instrument Setup
Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Wet
Standard Value? 0.400
Standard Solution #? 2023120
Sim Serial #? MP5318
Oper No? 133237

Flow Cal. Date: 06/21/2013
Slope 693
Intercept -721277

IR Calibration Date:	04/03/2024
	3um 9um
0th Coef(*100):	-20749 -22141
1st Coef(*100):	262599 135404
2nd Coef(*100):	2775 1304
H2O adj(mg/l*10k):	463 445

***** Test End *****

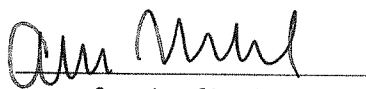
Operator Signature
ANNA NAREHOOD

Remarks:
Internal Print
Test
Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005951
Location = TOXL 8164.14.00 09/16
06/18/2025 10:01

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890- = |
ABCDEFGHIJKLMN0PQRSTUVWXYZ!@#%\$^&*()_+?

abcdefghijklmnopqrstuvwxy1234567890- = |
ABCDEFGHIJKLMN0PQRSTUVWXYZ!@#%\$^&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Wet
Standard Value? 0.400
Standard Solution #? 202312D
Sim Serial #? MP5318
Oper No? 133237

Flow Cal. Date: 06/21/2013
Slope 693
Intercept -721277

IR Calibration Date: 04/03/2024

	3um	9um
0th Coef(*100):	-20749	-22141
1st Coef(*100):	262599	135404
2nd Coef(*100):	2775	1304
H2O adj(mg/l*10k):	463	445

***** Printer Test End *****

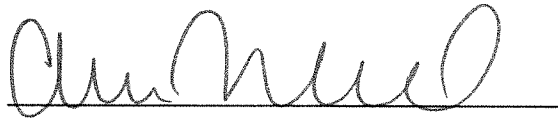

Operator Signature
ANNA NAREHOOD

Remarks: External Print Test

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005951
Location = TOXL 8164.14.00 09/16
06/18/2025 09:58

Test	AC	Time
01 Room Air	0.000	09:59
02 *Subject Test	REF*	10:00
03 Room Air	0.000	10:00

*Subject Test Refused

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = OTH Cit = REFUSAL
Dr. Lic. = ND/DIS821456
Simul Ser No = MP5318
Std Sol No = 202312D
County = 08 Oper No. = 133237

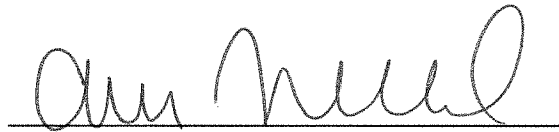

Operator Signature
ANNA NAREHOOD

Remarks:

Refusal den 6/18/25
Refusal

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005951
Location = TOXL 8164.14.00 09/16
06/18/2025 10:01

DIAGNOSTIC

Voltage/Current Test	Pass
RAM Test	Pass
EEPROM Checksum Test	Pass
Real Time Clock Test	Pass
DSP Test	Pass
Analytical Stability Test	Pass
Modem Test	Pass
Temp Regulation Test	Pass

Oper No. = 133237



Operator Signature
ANNA NAREHOOD

Remarks: *Diagnostic Check*

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005951
Location = TOXL 8164.14.00 09/16
06/18/2025 10:03

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:04
02 Std. Sol.	0.000	10:05
03 Room Air	0.000	10:05
04 Std. Sol.	0.000	10:06
05 Room Air	0.000	10:06
06 Std. Sol.	0.000	10:07
07 Room Air	0.000	10:08


08 Sim Temp = 34.0°C

Simul Ser No = MP3003

Std Sol No = NA

County = 08

Oper No. = 133237

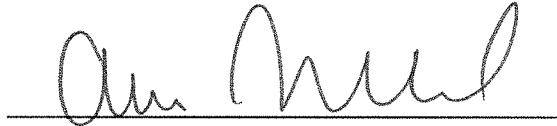

Operator Signature
ANNA NAREHOOD

Remarks: Zero AC check

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005951
Location = TOXL 8164.14.00 09/16
06/18/2025 10:17

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:18
02 Std. Sol.	0.020	10:19
03 Room Air	0.000	10:19
04 Std. Sol.	0.020	10:20
05 Room Air	0.000	10:21
06 Std. Sol.	0.020	10:21
07 Room Air	0.000	10:22


08 Sim Temp = 34.0°C

Simul Ser No = MP5321

Std Sol No = 202311D

County = 08

Oper No. = 133237

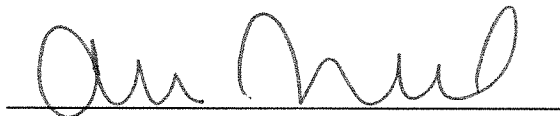

Operator Signature
ANNA NAREHOOD

Remarks: LOW AC CHECK - 0.020AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005951
Location = TOXL 8164.14.00 09/16
06/18/2025 10:23

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:24
02 Std. Sol.	0.290	10:25
03 Room Air	0.000	10:26
04 Std. Sol.	0.292	10:26
05 Room Air	0.000	10:27
06 Std. Sol.	0.292	10:28
07 Room Air	0.000	10:28


08 Sim Temp = 34.0°C

Simul Ser No = MP6035

Std Sol No = 202402C

County = 08

Oper No. = 133237

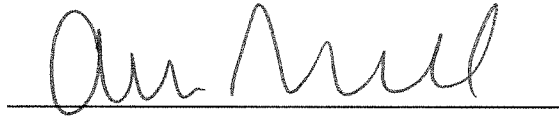

Operator Signature
ANNA NAREHOOD

Remarks: High AC check - 0.300 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005951
Location = TOXL 8164.14.00 09/16
06/18/2025 10:29

Test	AC	Time
01 Room Air	0.000	10:30
02 *Subject Test	INT*	10:31
03 Room Air	0.000	10:31

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = OTH Cit = INTERFERENT ICS
Dr. Lic. = ND/DIS821456
Lot No = 28423080A3
Cyl No = 41
Expiration Date = 11/05/2025
County = 08 Oper No. = 133237

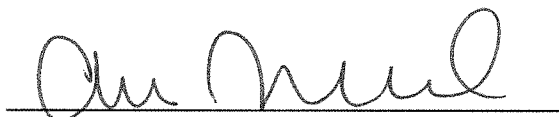

Operator Signature
ANNA NAREHOOD

Remarks: Interferent Check
0.10AC + 0.05% Acetone

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005951
Location = TOXL 8164.14.00 09/16
06/18/2025 10:32

Test	AC	Time
01 Room Air	0.000	10:32
02 *Subject Test	INT*	10:33
03 Room Air	0.000	10:33

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = OTH Cit = INTERFERENT ACE
Dr. Lic. = ND/DIS821456
Lot No = 28423080A3
Cyl No = 41
Expiration Date = 11/05/2025
County = 08 Oper No. = 133237



Operator Signature
ANNA NAREHOOD

Remarks: Interferent check
0.05% Acetone

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005951
Location = TOXL 8164.14.00 09/16
06/18/2025 10:33

Test	AC	Time
01 Room Air	0.000	10:34
02 *Subject Test	INT*	10:35
03 Room Air	0.000	10:35

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = OTH Cit = INTERFERENT ~~ACC~~ MeOH
Dr. Lic. = ND/DIS821456 AEN 6/18/25
Lot No = 28423080A3
Cyl No = 41
Expiration Date = 11/05/2025
County = 08 Oper No. = 133237

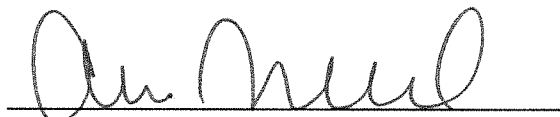

Operator Signature
ANNA NAREHOOD

Remarks: Interferent Check
0.10% Methanol

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005951
Location = TOXL 8164.14.00 09/16
06/18/2025 10:36

Test	AC	Time
01 Room Air	0.000	10:37
02 *Subject Test	INT*	10:37
03 Room Air	0.000	10:38

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = OTH Cit = INTERFERENT IPA
Dr. Lic. = ND/DIS821456
Lot No = 28423080A3
Cyl No = 41
Expiration Date = 11/05/2025
County = 08 Oper No. = 133237



Operator Signature
ANNA NAREHOOD

Remarks: Interferent check
0.10% Isopropanol

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005951
Location = TOXL 8164.14.00 09/16
06/18/2025 10:39

Test	AC	Time
01 Room Air	AMB*	10:40
02 Room Air	0.000	10:41

*Invalid Test
Check Ambient Conditions

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = OTH Cit = AMBIENT
Dr. Lic. = ND/DIS821456
Lot No = 28423080A3
Cyl No = 41
Expiration Date = 11/05/2025
County = 08 Oper No. = 133237

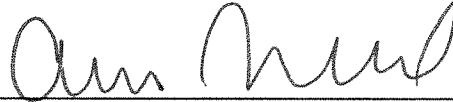

Operator Signature
ANNA NAREHOOD

Remarks: Ambient Fall

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005951
Location = TOXL 8164.14.00 09/16
06/18/2025 10:54

Test	AC	Time
01 Room Air	0.000	10:55
02 Invalid Sample	X.XXX	10:55
03 Room Air	0.000	10:56

*Invalid Test - Mouth Alcohol

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = OTH Cit = INVALID SAMPLE
Dr. Lic. = ND/DIS821456
Lot No = 28423080A3
Cyl No = 41
Expiration Date = 11/05/2025
County = 08 Oper No. = 133237



Operator Signature
ANNA NAREHOOD

Remarks: Invalid Sample

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005951
Location = TOXL 8164.14.00 09/16
06/18/2025 10:42

Test	AC	Time
01 Room Air	0.000	10:43
02 *Subject Test	ABT*	10:43
03 Room Air	0.000	10:44

*Sequence Aborted

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = OTH Cit = ABORTED SEQ
Dr. Lic. = ND/DIS821456
Lot No = 28423080A3
Cyl No = 41
Expiration Date = 11/05/2025
County = 08 Oper No. = 133237



Operator Signature
ANNA NAREHOOD

Remarks: Sequence Aborted

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005951
Location = TOXL 8164.14.00 09/16
06/18/2025 11:03

Test	AC	Time
01 Room Air	0.000	11:03
02 Subject Test 1	0.000	11:04
03 Room Air	0.000	11:04
04 Reported AC	0.000	11:04

No RFI Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = OTH Cit = ABA
Dr. Lic. = ND/DIS821456
Lot No = 28423080A3
Cyl No = 41
Expiration Date = 11/05/2025
County = 08 Oper No. = 133237

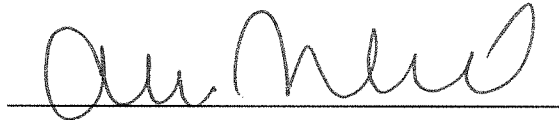

Operator Signature
ANNA NAREHOOD

Remarks: ABA Test

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005951
Location = TOXL 8164.14.00 09/16
06/18/2025 10:57

Test	AC	Time
01 Diagnostic	OK	10:58
02 Room Air	0.000	10:58
03 Subject Test 1	0.000	10:58
04 Room Air	0.000	11:01
05 Std. Gas	0.000*	11:02
06 Room Air	0.000	11:02

*Cal Check Out of Tolerance

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = OTH Cit = CAL CHECK OOT
Dr. Lic. = ND/DIS821456
Lot No = 28423080A3
Cyl No = 41
Expiration Date = 11/05/2025
County = 08 Oper No. = 133237

I followed the Approved Method and the
instructions displayed by the Intoxilyzer
in conducting this test.

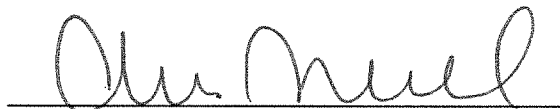

Operator Signature
ANNA NAREHOOD

Remarks: Calibration Check Out
of Tolerance

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005951
Location = TOXL 8164.14.00 09/16
06/18/2025 11:05

DRY CAL CHECK

Test	AC	Time
01 Room Air	RFI*	11:06
02 Room Air	0.000	11:06

*Invalid Test
Inhibited - RFI

Lot No = 28423080A3
Cyl No = 41
Exp Date = 11/05/2025
County = 08 Oper No. = 133237

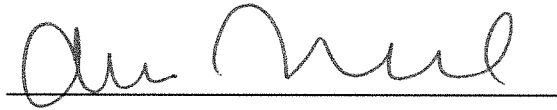

Operator Signature
ANNA NAREHOOD

Remarks: RFI check

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005951
Location = TOXL 8164.14.00 09/16
06/18/2025 11:08

Test	AC	Time
01 Diagnostic	OK	11:09
02 Room Air	0.000	11:09
03 *Subject Test	0.000*	11:12
04 Room Air	0.000	11:15
05 Std. Gas	0.075	11:16
06 Room Air	0.000	11:18
07 *Subject Test	0.000*	11:21
08 Room Air	0.000	11:22


No RFI Detected

*Deficient Sample - Value Printed was
Highest Obtained

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = OTH Cit = DEFICIENT SAMPL
Dr. Lic. = ND/DIS821456
Lot No = 28423080A3
Cyl No = 41
Expiration Date = 11/05/2025
County = 08 Oper No. = 133237

20 minute waiting period ascertained? Y
Clean Mouthpiece used and disposed? Y
Clean Mouthpiece used and disposed? Y

I followed the Approved Method and the
instructions displayed by the Intoxilyzer
in conducting this test.

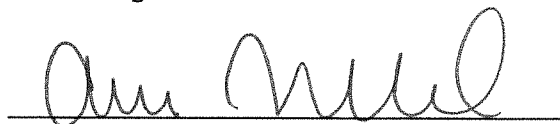

Operator Signature
ANNA NAREHOOD

Remarks: Deficient Sample

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005951
Location = TOXL 8164.14.00 09/16
06/18/2025 11:23

Test	AC	Time
01 Diagnostic	OK	11:23
02 Room Air	IPS*	11:24
03 Room Air	0.000	11:24

*Improper Sample

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = OTH Cit = IMPROPER SAMPLE
Dr. Lic. = ND/DIS821456
Lot No = 28423080A3
Cyl No = 41
Expiration Date = 11/05/2025
County = 08 Oper No. = 133237

I followed the Approved Method and the
instructions displayed by the Intoxilyzer
in conducting this test.



Operator Signature
ANNA NAREHOOD

Remarks: Improper Sample

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005951
Location = TOXL 8164.14.00 09/16
06/18/2025 11:25

Test	AC	Time
01 Diagnostic	OK	11:25
02 Room Air	0.000	11:26
03 Subject Test 1	0.000*	11:26
04 Room Air	0.000	11:29
05 Std. Gas	0.077	11:30
06 Room Air	0.000	11:31
07 Subject Test 2	0.095*	11:33
08 Room Air	0.000	11:34

*Difference Too Great

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = OTH Cit = DIFF TOO GREAT
Dr. Lic. = ND/DIS821456
Lot No = 28423080A3
Cyl No = 41
Expiration Date = 11/05/2025
County = 08 Oper No. = 133237

I followed the Approved Method and the
instructions displayed by the Intoxilyzer
in conducting this test.

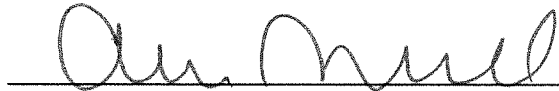

Operator Signature
ANNA NAREHOOD

Remarks: Difference Too Great

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005951
Location = TOXL 8164.14.00 09/16
06/18/2025 11:36

Test	AC	Time
01 Diagnostic	OK	11:37
02 Room Air	0.000	11:37
03 Subject Test 1	0.000	11:37
04 Room Air	0.000	11:40
05 Std. Gas	0.077	11:41
06 Room Air	0.000	11:42
07 Subject Test 2	0.000	11:42
08 Room Air	0.000	11:43
09 Reported AC	0.000	11:37

Difference OK
No RFI Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = OTH Cit = CMS TEST
Dr. Lic. = ND/DIS821456
Lot No = 28423080A3
Cyl No = 41
Expiration Date = 11/05/2025
County = 08 Oper No. = 133237

20 minute waiting period ascertained? Y
Clean Mouthpiece used and disposed? Y
Clean Mouthpiece used and disposed? Y

I followed the Approved Method and the
instructions displayed by the Intoxilyzer
in conducting this test.



Operator Signature
ANNA NAREHOOD

Remarks: CMS Test

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005951
Location = TOXL 8164.14.00 09/16
06/18/2025 11:44


Test	AC	Time
01 Room Air	0.000	11:45
02 Std. Gas	0.078	11:45
03 Room Air	0.000	11:45
04 Std. Gas	0.078	11:46
05 Room Air	0.000	11:46
06 Std. Gas	0.078	11:47
07 Room Air	0.000	11:47
08 Std. Gas	0.077	11:48
09 Room Air	0.000	11:48
10 Std. Gas	0.079	11:49
11 Room Air	0.000	11:49
12 Std. Gas	0.079	11:50
13 Room Air	0.000	11:50
14 Std. Gas	0.078	11:51
15 Room Air	0.000	11:51
16 Std. Gas	0.079	11:51
17 Room Air	0.000	11:52
18 Std. Gas	0.079	11:52
19 Room Air	0.000	11:53
20 Std. Gas	0.079	11:53
21 Room Air	0.000	11:54
22 Std. Gas	0.078	11:54
23 Room Air	0.000	11:55
24 Std. Gas	0.078	11:55
25 Room Air	0.000	11:56
26 Std. Gas	0.078	11:56
27 Room Air	0.000	11:57
28 Std. Gas	0.077	11:57
29 Room Air	0.000	11:58
30 Std. Gas	0.079	11:58
31 Room Air	0.000	11:59
32 Std. Gas	0.079	11:59
33 Room Air	0.000	11:59
34 Std. Gas	0.079	12:00
35 Room Air	0.000	12:00
36 Std. Gas	0.078	12:01
37 Room Air	0.000	12:01
38 Std. Gas	0.078	12:02
39 Room Air	0.000	12:02
40 Std. Gas	0.078	12:03
41 Room Air	0.000	12:03
42 Cal Check Stats:		
43 Average	0.0783	
44 Std Dev	0.0007	
45 Rel Std Dev	0.84%	

Operator Signature
ANNA NAREHOOD

Remarks:

Stability Test

Form 106-I8000

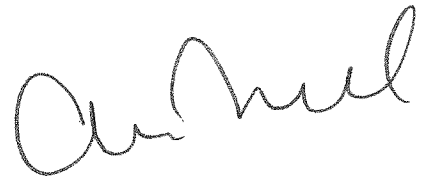A handwritten signature in black ink, appearing to read 'Anna Narehood', written in a cursive style.