



NORTH DAKOTA OFFICE OF ATTORNEY GENERAL  
CRIME LABORATORY DIVISION

INTOXILYZER® 8000 REPAIR AND MAINTENANCE

Intoxilyzer® 8000 Serial Number: 80-00 4943 Maintenance Location: TOXL

1. Reported Issue and Issue Found:

BROKEN modem port  
\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_

2. Repair/Maintenance Performed:

Modem port was repositioned and fixed.  
\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_

3. Does a Calibration Adjustment need to be performed? Yes or No

- If Yes, perform a calibration check.

☐ Configure dry gas standard (Level 1,S).

Known Value : \_\_\_\_\_ AC

Gas Cylinder Lot #: \_\_\_\_\_

Cylinder #: \_\_\_\_\_

Exp. Date: \_\_\_\_\_

☐ Dry Calibration Check (Level 1,C)

Test 1 \_\_\_\_\_ Test 2 \_\_\_\_\_ Test 3 \_\_\_\_\_

- If No, go to step 4.

4. Does the instrument flow sensor need to be adjusted? Yes or No


- If Yes, complete Flow Sensor Adjustment and Calibration (Document ID: 11872), then go to step 5.
- If No, go to step 5.

5. Does the instrument calibration need to be adjusted? Yes or No
- If Yes, complete Intoxilyzer® 8000 Calibration Adjustment (Document ID: 11859), then go to step 6.
  - If No, go to step 6.
6. Was the instrument cover removed for the repair? Yes or No
- If Yes, complete Intoxilyzer® 8000 Initial Inspection (Document ID: 11784), then go to step 7.
  - If No, go to step 7.
7. Is an Annual Inspection due on the Intoxilyzer® 8000? Yes or No
- If Yes, complete Intoxilyzer® 8000 Annual Inspection (Document ID: 11698), step 8 is not needed, and sign below.
  - If No, go to step 8.
8. Was an Intoxilyzer® 8000 Initial Inspection (Document ID: 11784) completed in step 6? Yes or No
- If Yes, no further action is needed and sign below.
  - If No, Complete Intoxilyzer® 8000 Calibration (Document ID: 11871) and sign below.

Remarks/Notes: N/A

Instrument is acceptable to be used in the field. Yes or No


If No, state reason(s) why:



Breath Alcohol Analyst Signature

19 June 2025

Date

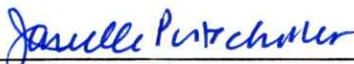

Reviewer Signature

20 June 2025

Date