

**ETHANOL GAS STANDARD CYLINDER REPORT**

NORTH DAKOTA OFFICE OF ATTORNEY GENERAL

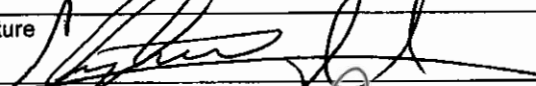
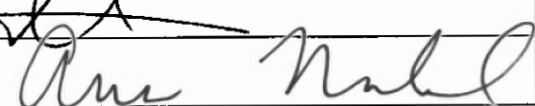
CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM

SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <b>HUARD, CHRISTOPHER</b>		
Location <b>WAHP RICHARD COUNTY JAIL, WAHPETON, ND</b>	Intoxilyzer® Serial Number <b>80-007086</b>	
Gas Lot Number <b>2842308DA3</b>	Gas Cylinder Number <b>047</b>	Gas Expiration Date <b>11/05/25</b>

## Check When Done:

- ☒ 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- ☒ 2. Perform an ACA Test (Level 1, Function C).  
Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
  - A. **0.078** AC
  - B. **0.078** AC
  - C. **0.078** AC
- ☒ 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- ☒ 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- ☒ 5. Send the Following to the Crime Laboratory:
  - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
  - B. ACA Test Record.

Chemical Test Operator Signature 	Date <b>06/26/25</b>
Reviewed By (Crime Laboratory Use Only) <b>Anna Narehood</b> 	Date <b>30June2025</b>


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-007086  
Location = WAHP      8164.16.00 09/18  
06/26/2025      14:39

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:39
02 Std. Gas	0.078	14:39
03 Room Air	0.000	14:40
04 Std. Gas	0.078	14:40
05 Room Air	0.000	14:41
06 Std. Gas	0.078	14:41
07 Room Air	0.000	14:42

Lot No = 28423080A3  
Cyl No = 47  
Exp Date = 11/05/2025  
County = 39      Oper No. = 132570

  
Operator Signature  
CHRISTOPHER HUARD

Remarks: GOOD A.C.A.

Form 106-I8000