



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
SFN 59281 (06/2018)

Serial Number 80-007086	Instrument Location WAHP RICHLAND COUNTY JAIL, WAHPETON, ND
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	


Check When Done:

- ☒ 1. Surge Protector Installed/Property Grounded.
- ☒ 2. Telephone Line Connected to Intoxilyzer® 8000.
- ☒ 3. Breath Tube Heated.
- ☒ 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- ☒ 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- ☒ 6. Run Tests:
 - ☒ A. Print Test (Level 1, Function P).
 - ☒ B. ACA Test (Level 1, Function C).
 - ☒ C. Radio Frequency Interference (RFI) Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- ☒ 7. Repair and/or Maintenance Performed (if any): _____
- ☒ 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- ☒ 9. File Previous Intoxilyzer® Record (SFN504096, Form 120-G) at the Intoxilyzer® Location at the Agency.
- ☒ 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature 	Date 06/26/25
--	-------------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified By Anna Narehood 	Certified Date 30 June 2025
--	---------------------------------------


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007086
Location = WAHP 8164.16.00 09/18
06/26/2025 14:39

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:39
02 Std. Gas	0.078	14:39
03 Room Air	0.000	14:40
04 Std. Gas	0.078	14:40
05 Room Air	0.000	14:41
06 Std. Gas	0.078	14:41
07 Room Air	0.000	14:42

Lot No = 28423080A3
Cyl No = 47
Exp Date = 11/05/2025
County = 39 Oper No. = 132570

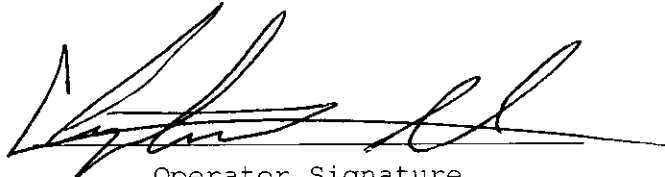
Operator Signature
CHRISTOPHER HUARD

Remarks: GOOD A.C.A.

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007086
Location = WAHP 8164.16.00 09/18
06/26/2025 14:43

Test	AC	Time
01 Room Air	0.000	14:46
02 Reference	RFI*	14:46
03 Room Air	0.000	14:46

*Invalid Test
Inhibited - RFI

Sub Name = DOE, JOHN NA
Sub DOB = 01/01/1987
Sub Sex = Male Weight = 210
Test = OTH Cit = 123456789
Dr. Lic. = ND/NDL123344
Lot No = 28423080A3
Cyl No = 47
Expiration Date = 11/05/2025
County = 39 Oper No. = 132570

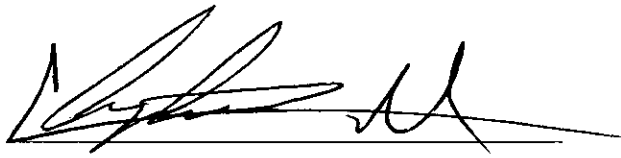

Operator Signature
CHRISTOPHER HUARD

Remarks: GOOD RFZ

Form 106-I8000