

**INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT**

NORTH DAKOTA OFFICE OF ATTORNEY GENERAL

CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM

SFN 59281 (06/2018)

| | |
|---|--|
| Serial Number 80-003066 | Instrument Location 1201 12th St SE Suite 1160 Bismarck ND |
| Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____ | |

Check When Done:

- ☒ 1. Surge Protector Installed/Property Grounded.
- ☒ 2. Telephone Line Connected to Intoxilyzer® 8000.
- ☒ 3. Breath Tube Heated.
- ☒ 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- ☒ 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- ☒ 6. Run Tests:
 - ☒ A. Print Test (Level 1, Function P).
 - ☒ B. ACA Test (Level 1, Function C).
 - ☒ C. Radio Frequency Interference (RFI) Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- ☐ 7. Repair and/or Maintenance Performed (if any): N/A
- ☒ 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- ☒ 9. File Previous Intoxilyzer® Record (SFN504096, Form 120-G) at the Intoxilyzer® Location at the Agency.
- ☒ 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

| | |
|-------------------------------|-----------------------|
| Field Inspector Signature | Date 7/6/25 |
|-------------------------------|-----------------------|

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

| | |
|---|---------------------------------------|
| Reviewed/Certified By Anna Narehood | Certified Date 15 July 2025 |
|---|---------------------------------------|

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003066
Location = MCSO 8164.14.00 09/16
07/06/2025 17:18

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#%^&*()_+?

Current Instrument Setup

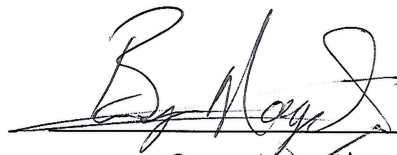
Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 01524080A4
Standard Cyl #? 17
Standard Expiration? 03/05/2026
Oper No? 130626

Flow Cal. Date: 07/29/2010
Slope 806
Intercept -895033

IR Calibration Date: 07/29/2010
3um 9um

| | | |
|--------------------|--------|--------|
| 0th Coef(*100): | -14075 | -5808 |
| 1st Coef(*100): | 272113 | 138204 |
| 2nd Coef(*100): | 1637 | 986 |
| H2O adj(mg/l*10k): | 289 | 148 |

***** Printer Test End *****



Operator Signature
BRYAN NAGEL

Remarks:

STATE OF NORTH DAKOTA } ss
COUNTY OF MCKENZIE

I certify that the foregoing is a true and correct
copy of the original document on file in this office.
Dated at Watford City, North Dakota.

on this 10th day of July 20, 25

Form 106-I8000

McKenzie County Sheriff's Department

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003066
Location = MCSO 8164.14.00 09/16
07/06/2025 17:22

DRY CAL CHECK

| Test | AC | Time |
|-------------|-------|-------|
| 01 Room Air | 0.000 | 17:23 |
| 02 Std. Gas | 0.078 | 17:23 |
| 03 Room Air | 0.000 | 17:24 |
| 04 Std. Gas | 0.079 | 17:24 |
| 05 Room Air | 0.000 | 17:24 |
| 06 Std. Gas | 0.079 | 17:25 |
| 07 Room Air | 0.000 | 17:25 |

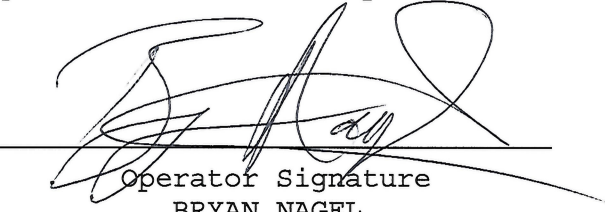
Lot No = 01524080A4

Cyl No = 17

Exp Date = 03/05/2026

County = 27

Oper No. = 130626

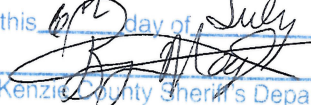

Operator Signature
BRYAN NAGEL

Remarks:

Form 106-I8000

STATE OF NORTH DAKOTA }
COUNTY OF MCKENZIE }^{ss}

I certify that the foregoing is a true and correct
copy of the original document on file in this office.
Dated at Watford City, North Dakota.

on this 17th day of July, 2025

McKenzie County Sheriff's Department

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003066
Location = MCSO 8164.14.00 09/16
07/06/2025 17:19

DRY CAL CHECK

| Test | AC | Time |
|-------------|-------|-------|
| 01 Room Air | 0.000 | 17:20 |
| 02 Std. Gas | 0.079 | 17:20 |
| 03 Room Air | RFI* | 17:20 |
| 04 Room Air | 0.000 | 17:21 |

*Invalid Test
Inhibited - RFI

Lot No = 01524080A4
Cyl No = 17
Exp Date = 03/05/2026
County = 27 Oper No. = 130626

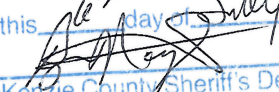

Operator Signature
BRYAN NAGEL

Remarks:

Form 106-I8000

STATE OF NORTH DAKOTA } ss
COUNTY OF MCKENZIE

I certify that the foregoing is a true and correct
copy of the original document on file in this office.
Dated at Watford City, North Dakota.

on this 16th day of July 2025

McKenzie County Sheriff's Department