



ETHANOL GAS STANDARD CYLINDER REPORT

NORTH DAKOTA OFFICE OF ATTORNEY GENERAL

CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM

SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <i>Beth Isaak</i>		
Location <i>DCSO</i>	Intoxilyzer® Serial Number <i>80-006498</i>	
Gas Lot Number <i>01524080A4</i>	Gas Cylinder Number <i>21</i>	Gas Expiration Date <i>3-5-26</i>

Check When Done:

- ☒ 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- ☒ 2. Perform an ACA Test (Level 1, Function C).
Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 - A. 0. *081* AC
 - B. 0. *080* AC
 - C. 0. *081* AC
- ☒ 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- ☒ 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- ☒ 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

Chemical Test Operator Signature <i>Beth Isaak</i>	Date <i>10-10-25</i>
Reviewed By (Crime Laboratory Use Only) Anna Narehood <i>Anna Narehood</i>	Date 27Oct2025

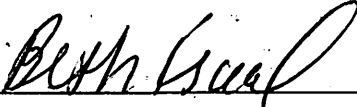
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006498
Location = DUNN 8164.14.00 09/16
10/10/2025 10:38

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:38
02 Std. Gas	0.081	10:39
03 Room Air	0.000	10:39
04 Std. Gas	0.080	10:40
05 Room Air	0.000	10:40
06 Std. Gas	0.081	10:41
07 Room Air	0.000	10:41

Lot No = 01524080A4
Cyl No = 21
Exp Date = 03/05/2026
County = 13 Oper No. = 131919


Operator Signature
BETH ISAAK

Remarks:

Form 106-I8000