



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
SFN 59281 (06/2018)

Serial Number 80-005957	Instrument Location Stanton
Reason for Install/Repair ☒ Install After Receiving From Crime Laboratory ☐ Install After Location Change ☐ Other (Specify) _____	

Check When Done:

- ☒ 1. Surge Protector Installed/Property Grounded.
☒ 2. Telephone Line Connected to Intoxilyzer® 8000.
☒ 3. Breath Tube Heated.
☒ 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
☒ 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
☒ 6. Run Tests:
☒ A. Print Test (Level 1, Function P).
☒ B. ACA Test (Level 1, Function C).
☒ C. Radio Frequency Interference (RFI) Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
☐ 7. Repair and/or Maintenance Performed (if any): _____
☒ 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
☒ 9. File Previous Intoxilyzer® Record (SFN504096, Form 120-G) at the Intoxilyzer® Location at the Agency.
☒ 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

AEN

10/29/25

Field Inspector Signature [Signature]	Date 10/28/25
---	-------------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified By Anna Narehood [Signature]	Certified Date 29Oct2025
--	------------------------------------

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005957
Location = STAT 8164.14.00 09/16
10/28/2025 13:40

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=|
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#%&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=|
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#%&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 25024080A3
Standard Cyl #? 59
Standard Expiration? 10/05/2026
Oper No? 120906

Flow Cal. Date: 06/21/2013
Slope 647
Intercept -613317

IR Calibration Date: 05/11/2015
 3um 9um

0th Coef(*100):	-16666	-18045
1st Coef(*100):	278478	132949
2nd Coef(*100):	3303	1397
H2O adj(mg/l*10k):	379	351

***** Printer Test End *****

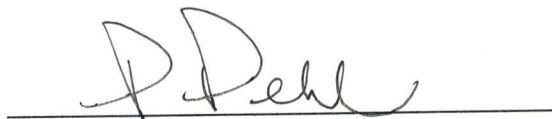

Operator Signature
PAMELA PEHL

Remarks:

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005957
Location = STAT 8164.14.00 09/16
10/28/2025 13:44

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:44
02 Std. Gas	0.079	13:45
03 Room Air	0.000	13:45
04 Std. Gas	0.079	13:45
05 Room Air	0.000	13:46
06 Std. Gas	0.078	13:46
07 Room Air	0.000	13:47

Lot No = 25024080A3
Cyl No = 59
Exp Date = 10/05/2026
County = 29 Oper No. = 120906



Operator Signature
PAMELA PEHL

Remarks:

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005957
Location = STAT 8164.14.00 09/16
10/28/2025 13:48

DRY CAL CHECK

Test	AC	Time
01 Room Air	RFI*	13:48
02 Room Air	RFI*	13:49

*Invalid Test
Inhibited - RFI

Lot No = 25024080A3
Cyl No = 59
Exp Date = 10/05/2026
County = 29 Oper No. = 120906



Operator Signature
PAMELA PEHL

Remarks:

Form 106-I8000