



# ETHANOL GAS STANDARD CYLINDER REPORT

NORTH DAKOTA OFFICE OF ATTORNEY GENERAL

CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM

SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <i>Tom Sharp</i>		
Location <i>Mandan</i>		Intoxilyzer® Serial Number <i>80-006506</i>
Gas Lot Number <i>25024080A3</i>	Gas Cylinder Number <i>10</i>	Gas Expiration Date <i>10/5/2026</i>

Check When Done:

- ☒ 1. Scan/Enter Gas Cylinder Information (Level 1, Function S). *Entered manually*
- ☒ 2. Perform an ACA Test (Level 1, Function C).  
Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).  
A. *0.080* AC  
B. *0.081* AC  
C. *0.081* AC
- ☒ 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- ☒ 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- ☒ 5. Send the Following to the Crime Laboratory:  
A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).  
B. ACA Test Record.

AEN  
6/11/25

Chemical Test Operator Signature <i>[Signature]</i>	Date <i>6/5/25</i>
Reviewed By (Crime Laboratory Use Only) Anna Narehood <i>[Signature]</i>	Date 11 June 2025

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006506  
Location = MAND      8164.14.00 09/16  
06/05/2025      09:30

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:30
02 Std. Gas	0.080	09:31
03 Room Air	0.000	09:31
04 Std. Gas	0.081	09:32
05 Room Air	0.000	09:32
06 Std. Gas	0.081	09:33
07 Room Air	0.000	09:33

Lot No = 25024080A3  
Cyl No = 10  
Exp Date = 10/05/2026  
County = 30      Oper No. = 020611



Operator Signature  
TOM SHARP

Remarks:

Form 106-I8000