



ETHANOL GAS STANDARD CYLINDER REPORT

NORTH DAKOTA OFFICE OF ATTORNEY GENERAL

CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM

SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <u>Mitch Slater</u>		
Location <u>Belcourt PD</u>		Intoxilyzer® Serial Number <u>80-005937</u>
Gas Lot Number <u>08425080A1</u>	Gas Cylinder Number <u>20</u>	Gas Expiration Date <u>5/5/2027</u>

Check When Done:

- ☒ 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- ☒ 2. Perform an ACA Test (Level 1, Function C).
Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 - A. 0.079 AC
 - B. 0.078 AC
 - C. 0.078 AC
- ☒ 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- ☒ 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- ☒ 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

Chemical Test Operator Signature <u>[Signature]</u>	Date <u>6-17-2025</u>
Reviewed By (Crime Laboratory Use Only) <u>Anna Narehood</u> <u>[Signature]</u>	Date 25June2025

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005937
Location = BELC 8164.14.00 09/16
06/17/2025 20:37

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	20:38
02 Std. Gas	0.079	20:38
03 Room Air	0.000	20:39
04 Std. Gas	0.078	20:39
05 Room Air	0.000	20:40
06 Std. Gas	0.078	20:40
07 Room Air	0.000	20:41

Lot No = 08425080A1
Cyl No = 20
Exp Date = 05/05/2027
County = 63 Oper No. = 130918



Operator Signature
MITCH SLATER

Remarks:

Form 106-I8000