

**ETHANOL GAS STANDARD CYLINDER REPORT**

NORTH DAKOTA OFFICE OF ATTORNEY GENERAL

CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM

SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <i>Thomas J Hiedeman</i>		
Location <i>Richland County Jail, Wahpeton, ND 58075</i>		Intoxilyzer® Serial Number <i>80-007086</i>
Gas Lot Number <i>1482508012</i>	Gas Cylinder Number <i>016</i>	Gas Expiration Date <i>09/05/2017</i>

Check When Done:

- ☒ 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- ☒ 2. Perform an ACA Test (Level 1, Function C).
Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
- A. *0.078* AC
- B. *0.077* AC
- C. *0.078* AC
- ☒ 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- ☒ 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- ☒ 5. Send the Following to the Crime Laboratory:
- A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
- B. ACA Test Record.

Chemical Test Operator Signature <i>Thomas J Hiedeman</i>	Date <i>11/03/2025</i>
Reviewed By (Crime Laboratory Use Only) Anna Ingemansen <i>Anna Ingemansen</i>	Date 12Nov2025

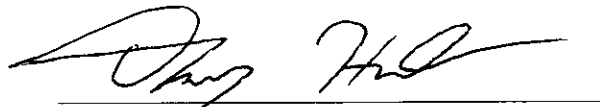
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007086
Location = WAHP 8164.16.00 09/18
11/03/2025 22:44

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	22:45
02 Std. Gas	0.078	22:45
03 Room Air	0.000	22:46
04 Std. Gas	0.077	22:46
05 Room Air	0.000	22:47
06 Std. Gas	0.078	22:47
07 Room Air	0.000	22:48

Lot No = 19825080A2
Cyl No = 16
Exp Date = 09/05/2027
County = 39 Oper No. = 133155


Operator Signature
THOMAS HIEDEMAN

Remarks: *ACA Test*

Form 106-I8000