



ETHANOL GAS STANDARD CYLINDER REPORT

NORTH DAKOTA OFFICE OF ATTORNEY GENERAL

CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM

SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <i>Tracy L. Meidinger</i>		
Location <i>UNDP</i>		Intoxilyzer® Serial Number <i>80-006688</i>
Gas Lot Number <i>19825080A2</i>	Gas Cylinder Number <i>31</i>	Gas Expiration Date <i>09-05-2027</i>

Check When Done:

- TLM* ☒ 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- TLM* ☒ 2. Perform an ACA Test (Level 1, Function C).
Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
A. *0.078* AC
B. *0.078* AC
C. *0.078* AC
- TLM* ☒ 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- TLM* ☒ 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- TLM* ☒ 5. Send the Following to the Crime Laboratory:
A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
B. ACA Test Record.

Chemical Test Operator Signature <i>Capt. Tracy L. Meidinger #258</i>	Date <i>11-10-2025</i>
Reviewed By (Crime Laboratory Use Only) <i>Anna Ingemansen</i>	Date <i>20Nov2025</i>


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006688
Location = UNDP 8164.14.00 09/16
11/10/2025 11:59

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:00
02 Std. Gas	0.078	12:00
03 Room Air	0.000	12:01
04 Std. Gas	0.078	12:01
05 Room Air	0.000	12:02
06 Std. Gas	0.078	12:02
07 Room Air	0.000	12:03

Lot No = 19825080A2
Cyl No = 31
Exp Date = 09/05/2027
County = 18 Oper No. = 119409


Operator Signature
TRACY MEIDINGER

Remarks:

Form 106-I8000