



ETHANOL GAS STANDARD CYLINDER REPORT

NORTH DAKOTA OFFICE OF ATTORNEY GENERAL

CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM

SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <i>Pamela Pehl</i>		
Location <i>Stanton</i>		Intoxilyzer® Serial Number <i>80-005944</i>
Gas Lot Number <i>19825080 A2</i>	Gas Cylinder Number <i>36</i>	Gas Expiration Date <i>09/05/2027</i>

Check When Done:

- ☒ 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- ☒ 2. Perform an ACA Test (Level 1, Function C).
Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 - A. *0.079* AC
 - B. *0.079* AC
 - C. *0.080* AC
- ☒ 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- ☒ 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- ☒ 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

Chemical Test Operator Signature <i>P Pehl</i>	Date <i>12/31/25</i>
Reviewed By (Crime Laboratory Use Only) Anna Ingemansen <i>Anna Ingemansen</i>	Date <i>05Jan2026</i>

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005944
Location = STAT 8164.14.00 09/16
12/31/2025 17:02

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	17:03
02 Std. Gas	0.079	17:03
03 Room Air	0.000	17:04
04 Std. Gas	0.079	17:04
05 Room Air	0.000	17:05
06 Std. Gas	0.080	17:05
07 Room Air	0.000	17:06

Lot No = 19825080A2
Cyl No = 36
Exp Date = 09/05/2027
County = 29 Oper No. = 120906



Operator Signature
PAMELA PEHL

Remarks:

Form 106-I8000