



**ETHANOL GAS STANDARD CYLINDER REPORT**  
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM  
 SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <i>Tyler Rispa</i>		
Location <i>Kenmare Police Department</i>		Intoxilyzer® Serial Number <i>80-004188</i>
Gas Lot Number <i>19825080A2</i>	Gas Cylinder Number <i>22</i>	Gas Expiration Date <i>09-05-2027</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).  
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
  - A. *0.081* AC
  - B. *0.081* AC
  - C. *0.081* AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
  - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
  - B. ACA Test Record.

Chemical Test Operator Signature <i>Tyler Rispa</i>	Date <i>03/02/2026</i>
Reviewed By (Crime Laboratory Use Only) Anna Ingemansen <i>Anna Ingemansen</i>	Date 05Mar2026

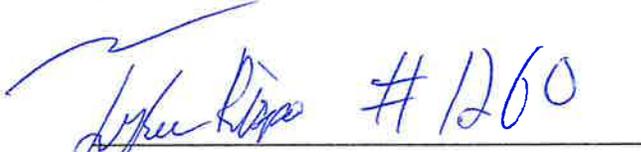
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004188  
Location = KMPD      8164.14.00 09/16  
03/02/2026      09:57

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:57
02 Std. Gas	0.081	09:58
03 Room Air	0.000	09:58
04 Std. Gas	0.081	09:59
05 Room Air	0.000	09:59
06 Std. Gas	0.081	10:00
07 Room Air	0.000	10:00

Lot No = 19825080A2  
Cyl No = 22  
Exp Date = 09/05/2027  
County = 51      Oper No. = 133682

 #1260  
Operator Signature  
TYLER RISPA

Remarks:

Form 106-I8000