



ETHANOL GAS STANDARD CYLINDER REPORT

NORTH DAKOTA OFFICE OF ATTORNEY GENERAL

CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM

SFN 59282 (08/2018)

Chemical Test Operator Name (Print) RILEIGH MULL		
Location Bldg. 339 GRAND FORKS AFB	Intoxilyzer® Serial Number 80-006509	
Gas Lot Number 26-4437	Gas Cylinder Number 106	Gas Expiration Date 1/29/2028

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 - A. 0.080 AC
 - B. 0.079 AC
 - C. 0.080 AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

Chemical Test Operator Signature <i>Riley Mull</i>	Date 5 MARCH 2026
Reviewed By (Crime Laboratory Use Only) Anna Ingemansen <i>Anna Ingemansen</i>	Date 20Mar2026


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006509
Location = GFAF 8164.14.00 09/16
03/05/2026 08:54

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	08:56
02 Std. Gas	0.080	08:56
03 Room Air	0.000	08:57
04 Std. Gas	0.079	08:57
05 Room Air	0.000	08:58
06 Std. Gas	0.080	08:58
07 Room Air	0.000	08:59

Lot No = 264437
Cyl No = 106
Exp Date = 01/29/2028
County = 65 Oper No. = 134011

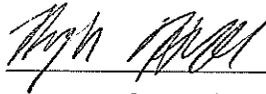

Operator Signature
RILEIGH HULL

Remarks:

Form 106-I8000