



ETHANOL GAS STANDARD CYLINDER REPORT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59282 (08/2018)

Chemical Test Operator Name (Print) Bryce Pottenger		
Location BOTT	Intoxilyzer® Serial Number 80-006673	
Gas Lot Number 26-4437	Gas Cylinder Number 119	Gas Expiration Date 1-29-2028

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 A. 0.079 AC
 B. 0.079 AC
 C. 0.079 AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 B. ACA Test Record.

Chemical Test Operator Signature <i>Bryce Pottenger</i>	Date 3-10-26
Reviewed By (Crime Laboratory Use Only) Anna Ingemansen <i>Anna Ingemansen</i>	Date 13Mar2026

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006673
Location = BOTT 8164.14.00 09/16
03/10/2026 18:04

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	18:05
02 Std. Gas	0.079	18:06
03 Room Air	0.000	18:06
04 Std. Gas	0.079	18:07
05 Room Air	0.000	18:07
06 Std. Gas	0.079	18:08
07 Room Air	0.000	18:08

Lot No = 264437
Cyl No = 0119
Exp Date = 01/29/2028
County = 05 Oper No. = 134061



Operator Signature
BRYCE POTTENGER

Remarks:

Form 106-I8000