



ETHANOL GAS STANDARD CYLINDER REPORT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59282 (08/2018)

Chemical Test Operator Name (Print) CHRISTOPHER ALMLIE		
Location BURKE SO	Intoxilyzer® Serial Number 80-C04947	
Gas Lot Number 26-4437	Gas Cylinder Number 114	Gas Expiration Date 1/29/2028

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 A. **0.078** AC
 B. **0.077** AC
 C. **0.078** AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

Chemical Test Operator Signature 	Date 3/10/2026
Reviewed By (Crime Laboratory Use Only) Anna Ingemansen	Date 13Mar2026

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004947
Location = BURK 8164.14.00 09/16
03/10/2026 15:48

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:49
02 Std. Gas	0.078	15:49
03 Room Air	0.000	15:50
04 Std. Gas	0.077	15:50
05 Room Air	0.000	15:51
06 Std. Gas	0.078	15:51
07 Room Air	0.000	15:52

Lot No = 264437
Cyl No = 114
Exp Date = 01/29/2028
County = 07 Oper No. = 132664


Operator Signature
CHRISTOPHER ALMLIE

Remarks:
good

Form 106-I8000