



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59281 (06/2018)

Serial Number	80-006500	Instrument Location	HARU
Reason for Install/Repair			
<input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____			

Check When Done:

- 1. Surge Protector Installed/Property Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. Radio Frequency Interference (RFI) Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if any): _____
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN504096, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature	<i>Kella Peterson</i>	Date	04/03/2026
---------------------------	-----------------------	------	------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified By	Anna Ingemansen	<i>Anna Ingemansen</i>	Certified Date	09April2026
-----------------------	-----------------	------------------------	----------------	-------------

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006500
Location = HARV 8164.14.00 09/16
04/03/2026 10:52

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:52
02 Std. Gas	0.082	10:53
03 Room Air	0.000	10:53
04 Std. Gas	0.080	10:53
05 Room Air	0.000	10:54
06 Std. Gas	0.081	10:54
07 Room Air	0.000	10:55

Lot No = 26 4437
Cyl No = 125
Exp Date = 01/29/2028
County = 52 Oper No. = 133723



Operator Signature
KELLA NORBY

Remarks: ACA Test

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006500
Location = HARV 8164.14.00 09/16
04/03/2026 10:57

Test	AC	Time
01 Room Air	RFI*	10:58
02 Room Air	RFI*	10:58

*Invalid Test
Inhibited - RFI

Sub Name = RFI, TEST TEST
Sub DOB = 01/01/2001
Sub Sex = Female Weight = NA
Test = DUI Cit = NA
Dr. Lic. = ND/NA
Lot No = 26 4437
Cyl No = 125
Expiration Date = 01/29/2028
County = 52 Oper No. = 133723

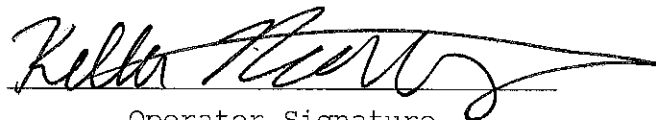

Operator Signature
KELLA NORBY

Remarks: RFI Test

Form 106-I8000