



# INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

NORTH DAKOTA OFFICE OF ATTORNEY GENERAL


CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM

SFN 59281 (06/2018)

Serial Number <b>80-005582</b>	Instrument Location <b>CROS</b>
Reason for Install/Repair	
<input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	


### Check When Done:

- 1. Surge Protector Installed/Property Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
  - A. Print Test (Level1, Function P).
  - B. ACA Test (Level 1, Function C).
  - C. Radio Frequency Interference (RFI) Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if any): **N/A**
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN504096, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature 	Date <b>04/07/25</b>
---	----------------------

### Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified By <b>Anna Ingemansen</b> 	Certified Date <b>22April2026</b>
---	-----------------------------------



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006682  
Location = CROS      8164.14.00 09/16  
04/06/2026      19:31

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	19:32
02 Std. Gas	0.079	19:32
03 Room Air	0.000	19:33
04 Std. Gas	0.079	19:33
05 Room Air	0.000	19:33
06 Std. Gas	0.079	19:34
07 Room Air	0.000	19:34


Lot No = 264437

Cyl No = 121

Exp Date = 01/29/2028

County = 12

Oper No. = 130831

  
\_\_\_\_\_  
Operator Signature  
MICHAEL DEHN

Remarks:

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006682  
Location = CROS      8164.14.00 09/16  
04/07/2026      09:35

Test	AC	Time
01 Room Air	RFI*	09:36
02 Room Air	RFI*	09:37

\*Invalid Test  
Inhibited - RFI

Sub Name = TEST, TEST TEST  
Sub DOB = 01/01/1999  
Sub Sex = Male      Weight = 200  
Test = DUI      Cit = 12345  
Dr. Lic. = ND/TES111234  
Lot No = 264437  
Cyl No = 121  
Expiration Date = 01/29/2028  
County = 12      Oper No. = 130831



Operator Signature  
MICHAEL DEHN

Remarks:

Form 106-I8000

~~The undersigned, having custody of the original record, certifies that the information herein is a true and correct copy of the original document as part of the...~~

The undersigned, having custody of the original record, certifies that the information herein is a true and correct copy of the original document as part of the...

      04/07/2026  
Signature      Date