



ETHANOL GAS STANDARD CYLINDER REPORT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <i>Jesse Hoff</i>		
Location <i>DKPD</i>	Intoxilyzer® Serial Number <i>80-004948</i>	
Gas Lot Number <i>26-4437</i>	Gas Cylinder Number <i>108</i>	Gas Expiration Date <i>01/29/2028</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 A. *0.081* AC
 B. *0.080* AC
 C. *0.080* AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

AEI 23Apr2026 Box #2 was checked by Anna Ingemansen. This does not impact the installation of the Ethanol Gas Standard.

Chemical Test Operator Signature <i>J Hoff</i>	Date <i>04/08/2026</i>
Reviewed By (Crime Laboratory Use Only) <i>Anna Ingemansen</i>	Date <i>23April2026</i>

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004948
Location = DKPD 8164.14.00 09/16
04/08/2026 14:26

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:28
02 Std. Gas	0.081	14:28
03 Room Air	0.000	14:28
04 Std. Gas	0.080	14:29
05 Room Air	0.000	14:29
06 Std. Gas	0.080	14:30
07 Room Air	0.000	14:30

Lot No = 264437
Cyl No = 108
Exp Date = 01/29/2028
County = 45 Oper No. = 130094



Operator Signature
JESSE HOFF

Remarks:

Form 106-I8000