



**INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT**  
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM  
 SFN 59281 (06/2018)

Serial Number <i>80-006506</i>	Instrument Location <i>MAND</i>
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Property Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
  - A. Print Test (Level 1, Function P).
  - B. ACA Test (Level 1, Function C).
  - C. Radio Frequency Interference (RFI) Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if any): \_\_\_\_\_
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN504096, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>[Signature]</i>	Date <i>4/8/2026</i>
---	-------------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified By <i>Anna Ingemansen</i>	<i>Anna Ingemansen</i>	Certified Date 27April2026
---	------------------------	-------------------------------

Number 6 was checked off by Anna Ingemansen. All tests were attached. This does not affect the instrument installation. AEI 27April2026



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006506  
Location = MAND                8164.14.00 09/16  
04/08/2026                        14:33

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:33
02 Std. Gas	0.082	14:34
03 Room Air	0.000	14:34
04 Std. Gas	0.081	14:34
05 Room Air	0.000	14:35
06 Std. Gas	0.082	14:35
07 Room Air	0.000	14:36

Lot No = 25024080A3  
Cyl No = 10  
Exp Date = 10/05/2026  
County = 30                      Oper No. = 020611

  
\_\_\_\_\_  
Operator Signature  
TOM SHARP

Remarks:

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006506  
Location = MAND                8164.14.00 09/16  
04/08/2026                        14:37

DRY CAL CHECK

Test	AC	Time
01 Room Air	RFI*	14:38
02 Room Air	RFI*	14:38

\*Invalid Test  
Inhibited - RFI

Lot No = 25024080A3  
Cyl No = 10  
Exp Date = 10/05/2026  
County = 30                      Oper No. = 020611

  
\_\_\_\_\_  
Operator Signature  
TOM SHARP

Remarks:

Form 106-I8000