



**INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT**  
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM  
 SFN 59281 (06/2018)

Serial Number <b>80-003061</b>	Instrument Location <b>Langdon</b>
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Property Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
  - A. Print Test (Level1, Function P).
  - B. ACA Test (Level 1, Function C).
  - C. Radio Frequency Interference (RFI) Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if any): N/A
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN504096, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature	Date <b>4-10-2026</b>
---------------------------	-----------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified By <b>Anna Ingemansen</b>	Certified Date <b>27 April 2026</b>
--	-------------------------------------

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-003061  
Location = LANG                    8164.14.00 09/16  
04/10/2026                            07:24

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890- = |  
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%&\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890- = |  
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%&\*()\_+?

Current Instrument Setup


Data Entry Mode:                    Enabled  
Start Test Sequence:                DABACABA  
Display Prelim Rslt?                Yes  
Display Third Digit?                Yes  
Inhib Printer(Y/N)?                 No  
Display Volume?                     No  
Disable On Memfull?                 Yes  
# of Print Copies?                   1  
Select Std (D/W/I)?                 Dry  
Standard Value?                     0.080  
Standard Lot #?                      264437  
Standard Cyl #?                      104  
Standard Expiration?                01/29/2028  
Oper No?                              021009

Flow Cal. Date:                     03/04/2011  
Slope                                 700  
Intercept                            -623726

IR Calibration Date:                04/10/2015  
    3um                                9um

-----  
0th Coef(\*100):                    -25722                            -17879  
1st Coef(\*100):                    266163                            136703  
2nd Coef(\*100):                    2302                                1151  
H2O adj(mg/l\*10k):                412                                 294

\*\*\*\*\* Printer Test End \*\*\*\*\*

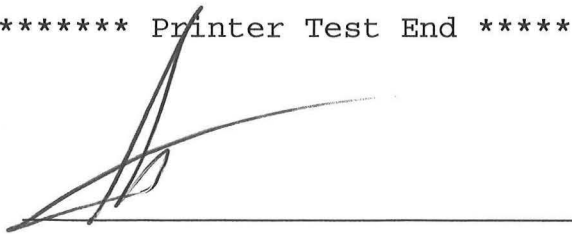

Operator Signature  
GABRIEL JOHNSTON

Remarks:

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003061  
Location = LANG      8164.14.00 09/16  
04/10/2026      09:04

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:05
02 Std. Gas	0.081	09:05
03 Room Air	0.000	09:06
04 Std. Gas	0.080	09:06
05 Room Air	0.000	09:07
06 Std. Gas	0.080	09:07
07 Room Air	0.000	09:07

Lot No = 25024080A3  
Cyl No = 008  
Exp Date = 10/05/2026  
County = 10      Oper No. = 021009

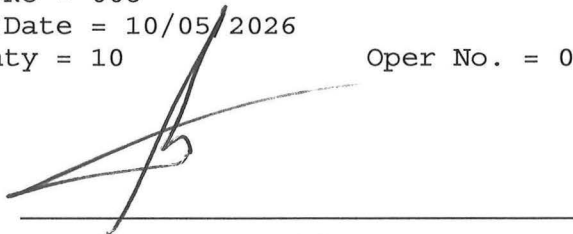

Operator Signature  
GABRIEL JOHNSTON

Remarks:

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-003061  
Location = LANG                    8164.14.00 09/16  
04/10/2026                            09:08

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:09
02 Std. Gas	0.081	09:09
03 Room Air	RFI*	09:10
04 Room Air	0.000	09:10

\*Invalid Test  
Inhibited - RFI

Lot No = 25024080A3  
Cyl No = 008  
Exp Date = 10/05/2026  
County = 10                            Oper No. = 021009

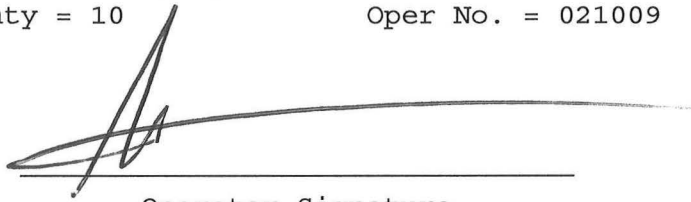

Operator Signature  
GABRIEL JOHNSTON

Remarks:

Form 106-I8000