



**INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT**  
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM  
 SFN 59281 (06/2018)

|   |                                    |
|---|------------------------------------|
| Serial Number<br><b>80-004936</b>   | Instrument Location<br><b>BMDL</b> |
| Reason for Install/Repair<br><input type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change<br><input checked="" type="checkbox"/> Other (Specify) <b>Install after Annual Inspection at Crime Lab</b> |                                    |

Check When Done:

- 1. Surge Protector Installed/Property Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
  - A. Print Test (Level 1, Function P).
  - B. ACA Test (Level 1, Function C).
  - C. Radio Frequency Interference (RFI) Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if any): **NA**
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN504096, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

|                               |                        |
|-------------------------------|------------------------|
| Field Inspector Signature<br> | Date<br><b>4/21/26</b> |
|-------------------------------|------------------------|

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

|   |  |
|---|--|
| Reviewed/Certified By<br><b>Anna Ingemansen</b> | Certified Date<br><b>30 April 2026</b> |
|---|--|



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004936  
Location = BMDC      8164.14.00 09/16  
04/21/2026      09:48

DRY CAL CHECK

| Test        | AC    | Time  |
|-------------|-------|-------|
| 01 Room Air | 0.000 | 09:49 |
| 02 Std. Gas | 0.078 | 09:49 |
| 03 Room Air | 0.000 | 09:50 |
| 04 Std. Gas | 0.078 | 09:50 |
| 05 Room Air | 0.000 | 09:51 |
| 06 Std. Gas | 0.078 | 09:51 |
| 07 Room Air | 0.000 | 09:52 |

Lot No = 264437  
Cyl No = 113  
Exp Date = 01/29/2028  
County = 08      Oper No. = 133598



---

Operator Signature  
DANIEL MAGELKY

Remarks:

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-004936  
Location = BMDC                8164.14.00 09/16  
04/21/2026                        09:54

| Test        | AC   | Time  |
|-------------|------|-------|
| 01 Room Air | RFI* | 09:56 |
| 02 Room Air | RFI* | 09:56 |

\*Invalid Test  
Inhibited - RFI

Sub Name = TEST, RFI NA  
Sub DOB = 01/01/2000  
Sub Sex = Unknown                Weight = NA  
Test = DUI                         Cit = NA  
Dr. Lic. = ND/NA  
Lot No = 264437  
Cyl No = 113  
Expiration Date = 01/29/2028  
County = 08                        Oper No. = 133598

  
Operator Signature  
DANIEL MAGELKY

Remarks:

Form 106-I8000