



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59281 (06/2018)

Serial Number	80-005944	Instrument Location	Stanton
Reason for Install/Repair			
<input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____			

Check When Done:

- 1. Surge Protector Installed/Property Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests: AEI 28April2026
 - A. Print Test (Level1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. Radio Frequency Interference (RFI) Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if any): _____
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN504096, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature	<i>P Dehl</i>	Date	4/22/26
---------------------------	---------------	------	---------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified By	Anna Ingemansen	<i>Anna Ingemansen</i>	Certified Date	28April2026
-----------------------	-----------------	------------------------	----------------	-------------

Checkbox 6 was marked off by Anna Ingemansen. All tests were printed and attached. This does not impact the installation of the instrument. AEI 28April2026

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005944
Location = STAT 8164.14.00 09/16
04/22/2026 14:45

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:46
02 Std. Gas	0.079	14:46
03 Room Air	0.000	14:47
04 Std. Gas	0.078	14:47
05 Room Air	0.000	14:48
06 Std. Gas	0.079	14:48
07 Room Air	0.000	14:49

Lot No = 19825080A2
Cyl No = 36
Exp Date = 09/05/2027
County = 29 Oper No. = 120906



Operator Signature
PAMELA PEHL

Remarks:

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005944
Location = STAT 8164.14.00 09/16
04/22/2026 14:49

DRY CAL CHECK

Test	AC	Time
01 Room Air	RFI*	14:49
02 Room Air	RFI*	14:49

*Invalid Test
Inhibited - RFI

Lot No = 19825080A2
Cyl No = 36
Exp Date = 09/05/2027
County = 29 Oper No. = 120906



Operator Signature
PAMELA PEHL

Remarks:

Form 106-I8000