



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59281 (06/2018)

Serial Number <i>80-004194</i>	Instrument Location <i>Killdeer PD</i>
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Property Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. Radio Frequency Interference (RFI) Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if any): *Yearly check at crime lab*
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN504096, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>Bvh Neal</i>	Date <i>4-23-26</i>
--	------------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified By Anna Ingemansen <i>Anna Ingemansen</i>	Certified Date 29 April 2026
--	---------------------------------

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004194
Location = KDPD 8164.14.00 09/16
04/23/2026 16:04

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:04
02 Std. Gas	0.080	16:05
03 Room Air	0.000	16:05
04 Std. Gas	0.078	16:06
05 Room Air	0.000	16:06
06 Std. Gas	0.079	16:07
07 Room Air	0.000	16:07

Lot No = 25024080A3
Cyl No = 2
Exp Date = 10/05/2026
County = 13 Oper No. = 131919



Operator Signature
BETH ISAAK

Remarks:

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004194
Location = KDPD 8164.14.00 09/16
04/23/2026 16:07

Test	AC	Time
01 Room Air	RFI*	16:09
02 Room Air	0.000	16:09

*Invalid Test
Inhibited - RFI

Sub Name = RFI TEST, RFI TEST NA
Sub DOB = 01/01/2026
Sub Sex = Unknown Weight = NA
Test = OTH Cit = NA
Dr. Lic. = ND/NA
Lot No = 25024080A3
Cyl No = 2
Expiration Date = 10/05/2026
County = 13 Oper No. = 131919



Operator Signature
BETH ISAAK

Remarks:

Form 106-I8000