



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59281 (06/2018)

Serial Number 80-005357	Instrument Location GRAF
Reason for Install/Repair	
<input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Property Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. Radio Frequency Interference (RFI) Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if any): _____
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN504096, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>Jacob Nibbo</i>	Date <i>5/8/26</i>
---	-----------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified By Anna Ingemansen <i>Ann Ingemansen</i>	Certified Date 13May2026
---	------------------------------------

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div.; Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005357
Location = GRAF 8164.14.00 09/16
05/08/2026 21:22

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	21:23
02 Std. Gas	0.081	21:23
03 Room Air	0.000	21:24
04 Std. Gas	0.081	21:24
05 Room Air	0.000	21:25
06 Std. Gas	0.081	21:25
07 Room Air	0.000	21:26

Lot No = 19825080A2
Cyl No = 26
Exp Date = 09/05/2027
County = 50 Oper No. = 132597

Jacob Nikko 4714

Operator Signature
JACOB NIKKO

Remarks:

Form 106-I8000

The Undersigned, having custody of the original public record, certifies that the information hereon is a true and correct copy of the original document maintained as a part of the activities of this agency.

Signature/Title *Jacob Nikko / Deputy*
Agency/Date *Walsh Co So / 5/8/26*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005357
Location = GRAF 8164.14.00 09/16
05/08/2026 21:27

Test	AC	Time
01 Room Air	RFI*	21:28
02 Room Air	0.000	21:29

*Invalid Test
Inhibited - RFI

Sub Name = TEST, RFI TEST
Sub DOB = 05/06/2000
Sub Sex = Male Weight = 180
Test = OTH Cit = 12345
Dr. Lic. = ND/123456789
Lot No = 19825080A2
Cyl No = 26
Expiration Date = 09/05/2027
County = 50 Oper No. = 132597


Operator Signature
JACOB NIKKO

Remarks:

Form 106-I8000

The Undersigned, having custody of the original public record, certifies that the information hereon is a true and correct copy of the original document maintained as a part of the activities of this agency.

Signature/Title Jacob Nikko / 5/8/26
Agency/Date Walsh Co So / Deputy