



**INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT**  
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM  
 SFN 59281 (06/2018)

Serial Number	80-007096	Instrument Location	DEVL
Reason for Install/Repair			
<input type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input checked="" type="checkbox"/> Other (Specify) <u>Install after Inspection</u>			

Check When Done:

- 1. Surge Protector Installed/Property Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
  - A. Print Test (Level 1, Function P).
  - B. ACA Test (Level 1, Function C).
  - C. Radio Frequency Interference (RFI) Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if any): N/A
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN504096, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature		Date	5/20/2026
---------------------------	--	------	-----------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified By	Anna Ingemansen		Certified Date	29May2026
-----------------------	-----------------	--	----------------	-----------




Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-007096  
Location = DEVL      8164.16.00 09/18  
05/20/2026      17:15

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	17:16
02 Std. Gas	0.079	17:16
03 Room Air	0.000	17:17
04 Std. Gas	0.080	17:17
05 Room Air	0.000	17:17
06 Std. Gas	0.079	17:18
07 Room Air	0.000	17:18

Lot No = 08425080A1  
Cyl No = 18  
Exp Date = 05/05/2027  
County = 36      Oper No. = 132008

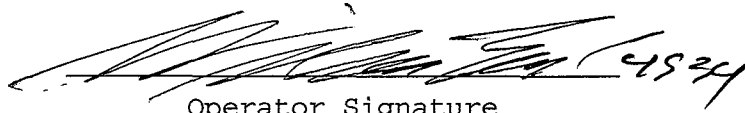  
Operator Signature  
WILLIAM EARL

Remarks:

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-007096  
Location = DEVL      8164.16.00 09/18  
05/20/2026      17:20

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	17:21
02 Std. Gas	0.079	17:21
03 Room Air	0.000	17:22
04 Std. Gas	RFI*	17:22
05 Room Air	0.000	17:23

\*Invalid Test  
Inhibited - RFI

Lot No = 08425080A1  
Cyl No = 18  
Exp Date = 05/05/2027  
County = 36      Oper No. = 132008

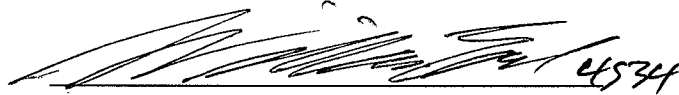

Operator Signature  
WILLIAM EARL

Remarks:

Form 106-I8000