



**INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT**  
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM  
 SFN 59281 (06/2018)

Serial Number <i>20-006687</i>	Instrument Location <i>Powers Lake Police Department</i>
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Property Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
  - A. Print Test (Level1, Function P).
  - B. ACA Test (Level 1, Function C).
  - C. Radio Frequency Interference (RFI) Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if any): *N/A*
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN504096, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

AEI  
6/3/26

Field Inspector Signature <i>Augusta Bou</i>	Date <i>5/31/26</i>
---	------------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified By <i>Anna Ingemansen</i>	Certified Date <i>03June2026</i>
---	-------------------------------------

Checklist #10 was checked off by Anna Ingemansen. These were all received by the Crime Lab. This does not affect the installation of the Instrument. 03June2026 AEI

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-006687  
Location = PLPD                    8164.14.00 09/16  
05/31/2026                            19:42

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890- = |  
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890- = |  
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&\*()\_+?

Current Instrument Setup

Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        Yes  
# of Print Copies?          1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            25024080A3  
Standard Cyl #?            55  
Standard Expiration?        10/05/2026  
Oper No?                    132545

Flow Cal. Date:            10/05/2018  
Slope                        696  
Intercept                   -609977

IR Calibration Date:        10/05/2018  
                                  3um                    9um

0th Coef(*100):	-11478	-18771
1st Coef(*100):	255256	132135
2nd Coef(*100):	4432	1676
H2O adj(mg/l*10k):	604	575

\*\*\*\*\* Printer Test End \*\*\*\*\*



Operator Signature  
AUSTIN HOWARD

Remarks:

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006687  
Location = PLPD      8164.14.00 09/16  
05/31/2026      19:43

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	19:43
02 Std. Gas	0.081	19:44
03 Room Air	0.000	19:44
04 Std. Gas	0.079	19:45
05 Room Air	0.000	19:45
06 Std. Gas	0.080	19:45
07 Room Air	0.000	19:46

Lot No = 25024080A3  
Cyl No = 55  
Exp Date = 10/05/2026  
County = 07      Oper No. = 132545



Operator Signature  
AUSTIN HOWARD

Remarks:

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006687  
Location = PLPD                    8164.14.00 09/16  
05/31/2026                            19:47

Test	AC	Time
01 Room Air	RFI*	19:50
02 Room Air	0.000	19:50

\*Invalid Test  
Inhibited - RFI

Sub Name = RFI TEST, INSTALL NA  
Sub DOB = 02/12/2000  
Sub Sex = Male                      Weight = 111  
Test = OTH                            Cit = NA  
Dr. Lic. = ND/NA  
Lot No = 25024080A3  
Cyl No = 55  
Expiration Date = 10/05/2026  
County = 07                            Oper No. = 132545



Operator Signature  
AUSTIN HOWARD

Remarks:

Form 106-I8000