



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
SFN 59281 (06/2018)

Serial Number 80-006513	Instrument Location Lisbon PD
Reason for Install/Repair ☒ Install After Receiving From Crime Laboratory ☐ Install After Location Change ☐ Other (Specify) _____	

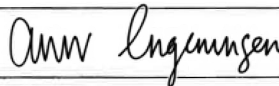
Check When Done:

- ☒ 1. Surge Protector Installed/Property Grounded.
- ☒ 2. Telephone Line Connected to Intoxilyzer® 8000.
- ☒ 3. Breath Tube Heated.
- ☒ 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- ☒ 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- ☒ 6. Run Tests:
 - ☒ A. Print Test (Level 1, Function P).
 - ☒ B. ACA Test (Level 1, Function C).
 - ☒ C. Radio Frequency Interference (RFI) Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- ☐ 7. Repair and/or Maintenance Performed (if any): _____
- ☒ 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- ☒ 9. File Previous Intoxilyzer® Record (SFN504096, Form 120-G) at the Intoxilyzer® Location at the Agency.
- ☒ 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature 	Date 04/23/2026
--	---------------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified By Anna Ingemansen 	Certified Date 14 July 2026
--	---------------------------------------

Checklist numbers 8-10 were checked off by Anna Ingemansen on 14 July 2026. Per email from officer Dylan Thomes, a new 120-G was placed at the instrument, the previous record was filed at the agency, and all forms were submitted. AEI 14 Jul 2026

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006513
Location = LISB 8164.14.00 09/16
04/23/2026 18:44

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	18:45
02 Std. Gas	0.081	18:46
03 Room Air	0.000	18:46
04 Std. Gas	0.081	18:46
05 Room Air	0.000	18:47
06 Std. Gas	0.082	18:47
07 Room Air	0.000	18:48

Lot No = 25024080A3
Cyl No = 53
Exp Date = 10/05/2026
County = 37 Oper No. = 133737



Operator Signature
DANNY GROSS

Remarks:

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006513
Location = LISB 8164.14.00 09/16
04/23/2026 18:49

Test	AC	Time
01 Room Air	RFI*	18:51
02 Room Air	RFI*	18:52

*Invalid Test
Inhibited - RFI

Sub Name = RFI TEST, TEST TEST
Sub DOB = 01/01/2000
Sub Sex = Unknown Weight = NA
Test = OTH Cit = NA
Dr. Lic. = ND/NA
Lot No = 25024080A3
Cyl No = 53
Expiration Date = 10/05/2026
County = 37 Oper No. = 133737



Operator Signature
DANNY GROSS

Remarks:

Form 106-I8000