



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
SFN 59281 (06/2018)

Serial Number 80-006503	Instrument Location Minot PD
Reason for Install/Repair ☒ Install After Receiving From Crime Laboratory ☐ Install After Location Change ☐ Other (Specify) _____	

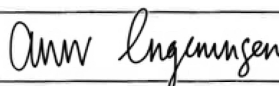
Check When Done:

- ☒ 1. Surge Protector Installed/Property Grounded.
- ☒ 2. Telephone Line Connected to Intoxilyzer® 8000.
- ☒ 3. Breath Tube Heated.
- ☒ 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- ☒ 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- ☒ 6. Run Tests:
 - ☒ A. Print Test (Level 1, Function P).
 - ☒ B. ACA Test (Level 1, Function C).
 - ☒ C. Radio Frequency Interference (RFI) Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- ☒ 7. Repair and/or Maintenance Performed (if any): _____
- ☒ 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- ☒ 9. File Previous Intoxilyzer® Record (SFN504096, Form 120-G) at the Intoxilyzer® Location at the Agency.
- ☒ 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature 	Date 5-22-2026
--	--------------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified By Anna Ingemansen 	Certified Date 16 July 2026
--	---------------------------------------

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006503
Location = MIPD 8164.14.00 09/16
04/29/2026 18:53

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 08425080A1
Standard Cyl #? 10
Standard Expiration? 05/05/2027
Oper No? 133426

Flow Cal. Date: 08/20/2015
Slope 663
Intercept -633300

IR Calibration Date: 06/12/2017
 3um 9um

0th Coef(*100):	-6741	-15632
1st Coef(*100):	256583	132194
2nd Coef(*100):	3028	1131
H2O adj(mg/l*10k):	354	358

***** Printer Test End *****


Operator Signature
CODY BAKER

Remarks:

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006503
Location = MIPD 8164.14.00 09/16
05/22/2026 10:49

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:49
02 Std. Gas	0.080	10:50
03 Room Air	0.000	10:50
04 Std. Gas	0.080	10:51
05 Room Air	0.000	10:51
06 Std. Gas	0.080	10:52
07 Room Air	0.000	10:52

Lot No = 264437
Cyl No = 0116
Exp Date = 01/29/2028
County = 64 Oper No. = 133426


Operator Signature
CODY BAKER

Remarks:

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006503
Location = MIPD 8164.14.00 09/16
05/22/2026 10:56

Test	AC	Time
01 Room Air	RFI*	10:57
02 Room Air	RFI*	10:57

*Invalid Test
Inhibited - RFI

Sub Name = BAKER, CODY RYAN
Sub DOB = 01/19/1995
Sub Sex = Male Weight = NA
Test = DUI Cit = NA
Dr. Lic. = ND/NDL180965
Lot No = 264437
Cyl No = 0116
Expiration Date = 01/29/2028
County = 64 Oper No. = 133426



Operator Signature
CODY BAKER

Remarks:

Form 106-I8000