



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
SFN 59281 (06/2018)

Serial Number 80-004948	Instrument Location DKPD
Reason for Install/Repair ☒ Install After Receiving From Crime Laboratory ☐ Install After Location Change ☐ Other (Specify) _____	


Check When Done:

- ☒ 1. Surge Protector Installed/Property Grounded.
- ☒ 2. Telephone Line Connected to Intoxilyzer® 8000.
- ☒ 3. Breath Tube Heated.
- ☒ 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- ☒ 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- ☒ 6. Run Tests:
 - ☒ A. Print Test (Level1, Function P).
 - ☒ B. ACA Test (Level 1, Function C).
 - ☒ C. Radio Frequency Interference (RFI) Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- ☒ 7. Repair and/or Maintenance Performed (if any): operator file updated
- ☒ 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- ☒ 9. File Previous Intoxilyzer® Record (SFN504096, Form 120-G) at the Intoxilyzer® Location at the Agency.
- ☒ 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature 	Date 8/5/2026
--	------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

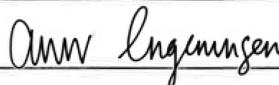Reviewed/Certified By Anna Ingemansen 	Certified Date 07Aug2026
---	-----------------------------

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004948
Location = DKPD 8164.14.00 09/16
08/05/2026 13:45

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:46
02 Std. Gas	0.081	13:47
03 Room Air	0.000	13:47
04 Std. Gas	0.080	13:48
05 Room Air	0.000	13:48
06 Std. Gas	0.080	13:48
07 Room Air	0.000	13:49

Lot No = 264437
Cyl No = 108
Exp Date = 01/29/2028
County = 45 Oper No. = 130094



Operator Signature
JESSE HOFF

Remarks:


Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004948
Location = DKPD 8164.14.00 09/16
08/05/2026 13:40

Test	AC	Time
01 Room Air	0.000	13:44
02 *Subject Test	RFI*	13:44
03 Room Air	0.000	13:44

*Invalid Test
Inhibited - RFI

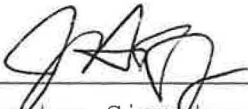

Operator Signature
JESSE HOFF

Remarks:

Form 106-I8000