

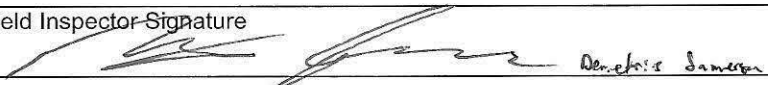


INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
SFN 59281 (06/2018)

Serial Number 80-004189	Instrument Location NTPD
Reason for Install/Repair ☐ Install After Receiving From Crime Laboratory ☒ Install After Location Change ☐ Other (Specify) _____	

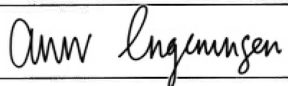
Check When Done:

- ☒ 1. Surge Protector Installed/Property Grounded.
- ☒ 2. Telephone Line Connected to Intoxilyzer® 8000.
- ☒ 3. Breath Tube Heated.
- ☒ 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- ☒ 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- ☒ 6. Run Tests:
 - ☒ A. Print Test (Level 1, Function P).
 - ☒ B. ACA Test (Level 1, Function C).
 - ☒ C. Radio Frequency Interference (RFI) Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- ☒ 7. Repair and/or Maintenance Performed (if any): None
- ☒ 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- ☒ 9. File Previous Intoxilyzer® Record (SFN504096, Form 120-G) at the Intoxilyzer® Location at the Agency.
- ☒ 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature  Detective Samson	Date 9-2-2026
--	-------------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified By Anna Ingemansen 	Certified Date 02Sept2026
---	-------------------------------------

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004189
Location = NTPD 8164.14.00 09/16
09/02/2026 12:29

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#%&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#%&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 25024080A3
Standard Cyl #? 003
Standard Expiration? 10/05/2026
Oper No? 132358

Flow Cal. Date: 02/02/2011
Slope 628
Intercept -452608

IR Calibration Date: 05/19/2016
 3um 9um

0th Coef(*100):	-17767	-18761
1st Coef(*100):	263439	133606
2nd Coef(*100):	1960	1140
H2O adj(mg/l*10k):	341	362

***** Printer Test End *****



Operator Signature
DEMETRIUS JAMERSON

Remarks:

Print test

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004189
Location = NTPD 8164.14.00 09/16
09/02/2026 12:35

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:35
02 Std. Gas	0.080	12:36
03 Room Air	0.000	12:36
04 Std. Gas	0.079	12:36
05 Room Air	0.000	12:37
06 Std. Gas	0.079	12:37
07 Room Air	0.000	12:38

Lot No = 25024080A3
Cyl No = 003
Exp Date = 10/05/2026
County = 31 Oper No. = 132358


Operator Signature
DEMETRIUS JAMERSON

Remarks:

ACA test

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

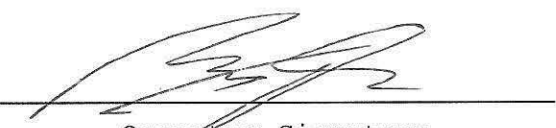
CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004189
Location = NTPD 8164.14.00 09/16
09/02/2026 12:38

DRY CAL CHECK

Test	AC	Time
01 Room Air	RFI*	12:39
02 Room Air	0.000	12:39

*Invalid Test
Inhibited - RFI

Lot No = 25024080A3
Cyl No = 003
Exp Date = 10/05/2026
County = 31 Oper No. = 132358


Operator Signature
DEMETRIUS JAMERSON

Remarks:

RFI test

Form 106-I8000